Nasaleze Allergy Blocker is a preventative treatment in the fight against hay fever and allergies. Nasaleze Allergy Blocker has been clinically proven to reduce sneezing, runny nose and sinus congestion.

As such, Nasaleze Allergy Blocker should be used before coming into contact with allergens to prevent them from landing on the nasal membrane and starting the allergic response.

Unlike medicines that help relieve allergy symptoms, Nasaleze Allergy Blocker works preventatively to stop an allergic reaction before symptoms start. Nasaleze Allergy Blocker is clinically proven to work effectively in mild to moderate allergic rhinitis sufferers.

Nasaleze Allergy Blocker is a powder nasal spray. The powder turns to a gel in the nose which creates a barrier to shield and protect from airborne allergens.

To defend against hay fever and allergies, Nasaleze Allergy Blocker needs to be applied 2 to 3 times per day to maintain an effective barrier. Nasaleze Allergy Blocker should last 6 hours per application.

Nasaleze Allergy Blocker is suitable for adults, athletes, pregnant and breast-feeding women and children from 18 months.

For more information visit: [www.nasaleze.com](http://www.nasaleze.com)

Please read enclosed leaflet carefully for complete product information.

Ingredients: inert natural cellulose powder and peppermint powder.

Keep in a cool dry place. Avoid contact with eyes. Once opened use within 6 months.

Nasaleze

Nasaleze

Nasaleze

SPRAY

TRAP & PROTECT

HELPS DEFEND AGAINST HAY FEVER & ALLERGIES

ALLERGY BLOCKER

STOP HAY FEVER & ALLERGIES BEFORE THEY START

2 MINUTES

POWDER SPRAY CLINICALLY PROVEN

Manufactured by Nasaleze Ltd, Unit 8, Woodside House, Badfletcher Business Park, Douglas, Isle of Man, IM4 4DE, UK

ES 177

Atlantic Bridge Ltd, 19 Willowside, Woodley, Berkshire, RG5 4HJ, UK

Distributed by Nasaleze International Ltd, Nunsey Hill, Old Castleway Road, Douglas, IM2 1QA, Isle of Man

Medicines & Healthcare Products Regulatory Agency Ref. No CA007190

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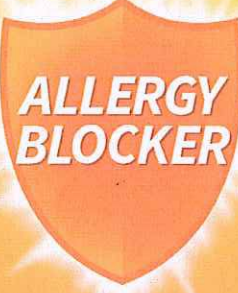
Địa chỉ: Unit 3, Woodside House, Ballafletcher Business Park, Douglas, Isle of Man, IM4 4QE, UK

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**ALLERGY  
BLOCKER**

**800mg**

**Read leaflet before first use.**  
**Directions:** Blow your nose.  
Apply 1 to 2 sprays into each nostril – Inhale gently while squeezing. Re-apply each time you blow your nose.

*Keep in a cool dry place.  
Avoid contact with eyes.  
Once opened use within 6 months.*

Nasa-ALL 11/18

**LOT**

EC REP Atlantic Bridge Ltd.  
19 Willowside,  
Woodley, Berkshire,  
RG5 4HJ, UK

Nasaleze Ltd. Unit 3,  
Woodside House,  
Ballafletcher Business  
Park, Douglas, Isle of  
Man, IM4 4QE, UK

Nasaleze Allergy Blocker is a preventative treatment in the fight against hay fever and allergies. Nasaleze Allergy Blocker has been clinically proven to reduce sneezing, runny nose and sinus congestion.

As such, Nasaleze Allergy Blocker should be used before coming into contact with allergens to prevent them from landing on the nasal membrane and starting the allergic response.

Unlike medicines that help relieve allergy symptoms, Nasaleze Allergy Blocker works preventatively to stop an allergic reaction before symptoms start. Nasaleze Allergy Blocker is clinically proven to work effectively in mild to moderate allergic rhinitis sufferers.

Nasaleze Allergy Blocker is a powder nasal spray. The powder turns to a gel in the nose which creates a barrier to shield and protect from airborne allergens.

To defend against hay fever and allergies, Nasaleze Allergy Blocker needs to be applied 2 to 3 times per day to maintain an effective barrier. Nasaleze Allergy Blocker should last 6 hours per application.

Nasaleze Allergy Blocker is suitable for children from 18 months.

For more information visit: [www.nasaleze.com](http://www.nasaleze.com)

Please read enclosed leaflet carefully for complete product information.

Ingredients: inert natural cellulose powder and strawberry powder.

Keep in a cool dry place.

Avoid contact with eyes.

Once opened use within 6 months.

Nasaleze

Nasaleze

Nasaleze

SPRAY

TRAP & PROTECT

HELPS DEFEND AGAINST HAY FEVER & ALLERGIES

Children's ALLERGY BLOCKER

ACTS TOGETHER 2 MINUTES

POWDER SPRAY CLINICALLY PROVEN

Manufactured by Nasaleze Ltd, Unit 9, Woodside House, Ballsbridge Business Park, Douglas, Isle of Man, IM4 4QE, UK

Atlantic Bridge Ltd, 19 Willowside, Woodley, Berkshire, RG5 4HL, UK

Distributed by Nasaleze International Ltd, Nunnery Mills, 201 Castlestown Road, Douglas, IM2 1QA, Isle of Man

Medicines & Healthcare Products Regulatory Agency Ref. No CA007180

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**800mg**

**CE**

**Read leaflet before first use.**  
**Directions:** Blow your nose.  
Apply 1 to 2 sprays into each nostril – Inhale gently while squeezing. Re-apply each time you blow your nose.  
*Keep in a cool dry place.  
Avoid contact with eyes.  
Once opened use within 6 months.*

Nasa-CHILD 11/18

**LOT**

Atlantic Bridge Ltd.  
19 Willowside,  
Woodley, Berkshire,  
RG5 4HJ, UK

EC REP

Nasaleze Ltd. Unit 3,  
Woodside House,  
Ballafletcher Business  
Park, Douglas, Isle of  
Man, IM4 4QE, UK

**COLD & FLU BLOCKER**

**Nasaleze**

Nasaleze Cold & Flu Blocker is a preventative treatment in the fight against cold and flu viruses. Nasaleze Cold & Flu Blocker has been clinically proven to help stop the onset of a cold or flu or shorten its duration.

Nasaleze traps and deactivates airborne germs and viruses to help stop the symptoms of cold and flu taking hold.

Unlike medicines that help relieve cold and flu symptoms, Nasaleze works preventatively to stop a cold or flu before symptoms start. Nasaleze is clinically proven to work effectively in adults and children of all ages.

Nasaleze Cold & Flu Blocker is a powder nasal spray. The powder turns to a gel in the nose which creates a barrier to shield and protect from airborne germs and viruses.

To defend against cold and flu, Nasaleze Cold & Flu Blocker needs to be applied 2 to 3 times per day.

Nasaleze Cold & Flu Blocker is suitable for adults, athletes, pregnant and breast-feeding women and children from 3 years.

For more information visit: [www.nasaleze.com](http://www.nasaleze.com)

Please read enclosed leaflet carefully for complete product information.

Ingredients: Inert natural cellulose powder, peppermint powder and garlic.

Keep in a cool dry place. Avoid contact with eyes. Once opened use within 6 months.

**Nasaleze**

SPRAY

TRAP & DEACTIVATE

HELPS DEFEND AGAINST COLD & FLU

**Nasaleze**

**COLD & FLU BLOCKER**

STARTS TO DEFEND IN 2 MINUTES

POWDER SPRAY CLINICALLY PROVEN

**Nasaleze**

Manufactured by Nasaleze Ltd, Unit 3, Woodside House, Bullfletcher Business Park, Douglas, Isle of Man, IM4 4DE, UK

Atlantic Bridge Ltd, 19 Willowside, Woodley, Berkshire, RG5 4HU, UK

Distributed by Nasaleze International Ltd, Nunmery Mills, Old Castletown Road, Douglas, IM2 1QA, Isle of Man

Medicines & Healthcare Products Regulatory Agency Ref. No CA007190

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The image shows the packaging label for Nasaleze Cold & Flu Blocker. The label is blue and white. On the left, there is a factory icon and text: "Nasaleze Ltd. Unit 3, Woodside House, Ballafletcher Business Park, Douglas, Isle of Man, IM4 4QE, UK". Below this is the "EC REP" logo and text: "Atlantic Bridge Ltd. 19 Willowside, Woodley, Berkshire, RG5 4HJ, UK". Further down is the "LOT" logo and an hourglass icon. In the center, the "Nasaleze" logo is at the top, followed by a shield icon containing the text "COLD & FLU BLOCKER". To the right of the shield, it says "800mg" and the CE mark. Below the CE mark is an umbrella icon. On the far right, there is a vertical text "Nasa-COLD 11/18". At the bottom right, there is a red circular stamp with the text "M.S.D.N. 0315981074-C.T.T.N.H.H. CÔNG TY TNHH SV HEALTHCARE THÀNH PHỐ HỒ CHÍ MINH".

SHIELD  
PM

Nasaleze

Nasaleze PM Shield is a preventative treatment in the fight against modern pollution. Particulate matter (PM) can be made up of many different chemicals.

PM can't be seen, smelled or tasted – but these particles, especially the smaller ones (PM10 or less), can be very harmful to the human body.

Nasaleze PM Shield has been proven to reduce diffusion of PM2.5 particles. As these are very small particles protection is also given against larger ones, such as PM10.

Nasaleze PM Shield is a powder nasal spray. The powder turns to a gel in the nose which creates a barrier to shield and protect the nasal membrane against particulate matter.

To defend against pollution, Nasaleze PM Shield needs to be applied 2 to 3 times per day. Nasaleze PM Shield should last 6 hours per application.

Nasaleze PM Shield is suitable for adults, athletes, pregnant and breast-feeding women and children from 18 months.

For more information visit:  
[www.nasaleze.com](http://www.nasaleze.com)

Please read enclosed leaflet carefully for complete product information.

Ingredients: Inert natural cellulose powder and lemon.

Keep in a cool dry place.  
Avoid contact with eyes.  
Once opened use within 6 months.

Nasaleze

SPRAY

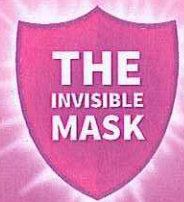
TRAP & PROTECT

HELPS DEFEND AGAINST  
PM & POLLUTION

Nasaleze

PM  
SHIELD

STARTS TO DEFEND IN  
2  
MINUTES

POWDER SPRAY  
CLINICALLY PROVEN

LOT

Nasa-PM 11/18

Nasaleze

Manufactured by Nasaleze Ltd, Unit 3, Woodside House, Ballalatcher Business Park, Douglas, Isle of Man, IM4 4GE, UK

CE

Atlantic Bridge Ltd, 18 Willowside, Woodley, Berkshire, RG5 4HL, UK

Distributed by Nasaleze International Ltd, Nunery Mills, Old Castletown Road, Douglas, IM2 1QA, Isle of Man

Medicines & Healthcare Products Regulatory Agency Ref. No CA007190

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Nasaleze Ltd. Unit 3,  
Woodside House,  
Ballafletcher Business  
Park, Douglas, Isle of  
Man, IM4 4QE, UK

EC REP Atlantic Bridge Ltd.  
19 Willowside,  
Woodley, Berkshire,  
RG5 4HJ, UK

LOT

**Nasaleze®**

**PM  
SHIELD**

**800mg**

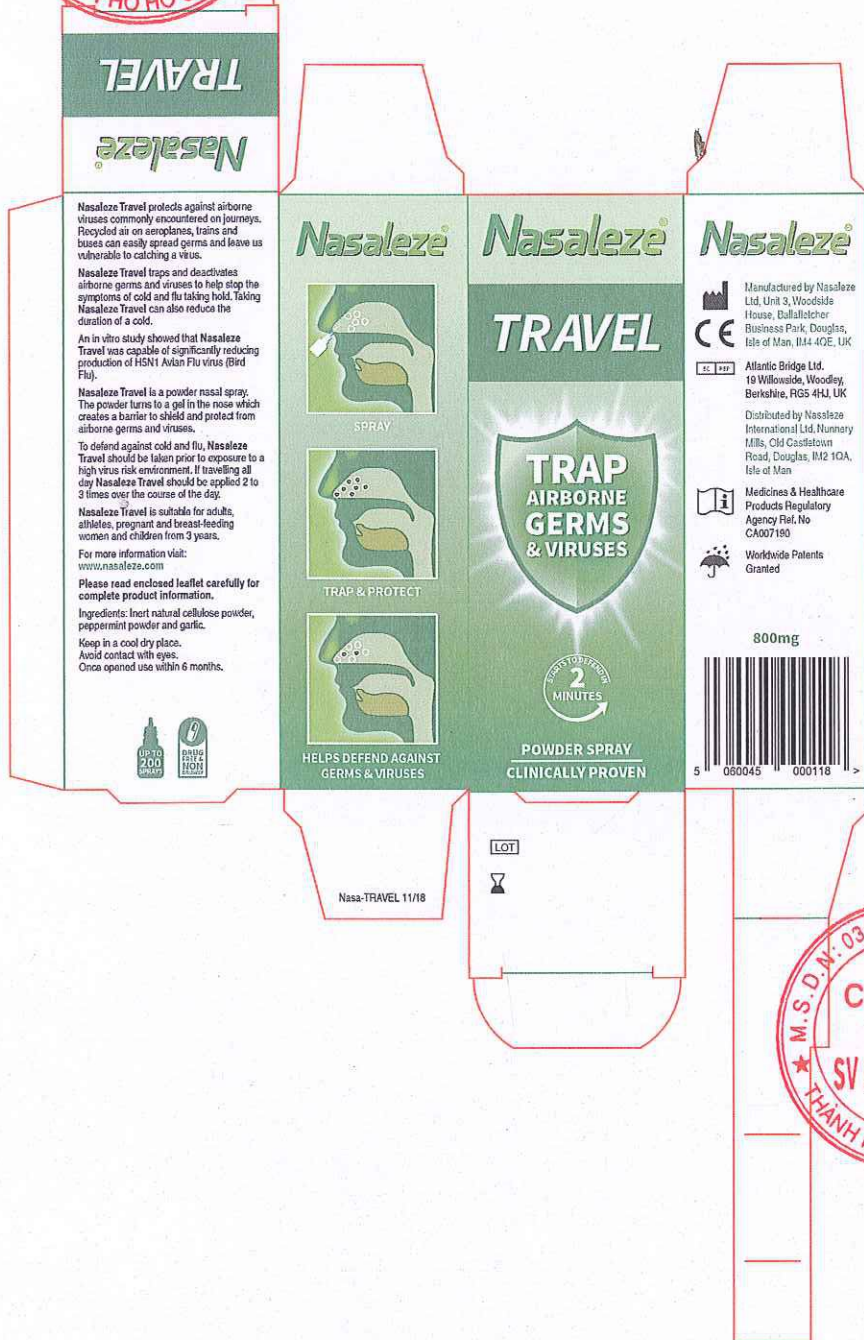
CE

Read leaflet before first use.  
**Directions:** Blow your nose.  
Apply 1 to 2 sprays into each  
nostril – Inhale gently while  
squeezing. Re-apply each time  
you blow your nose.

*Keep in a cool dry place.  
Avoid contact with eyes.  
Once opened use within 6  
months.*

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Nasaleze Travel protects against airborne viruses commonly encountered on journeys. Recycled air on aeroplanes, trains and buses can easily spread germs and leave us vulnerable to catching a virus.

Nasaleze Travel traps and deactivates airborne germs and viruses to help stop the symptoms of cold and flu taking hold. Taking Nasaleze Travel can also reduce the duration of a cold.

An in vitro study showed that Nasaleze Travel was capable of significantly reducing production of H5N1 Avian Flu virus (Bird Flu).

Nasaleze Travel is a powder nasal spray. The powder turns to a gel in the nose which creates a barrier to shield and protect from airborne germs and viruses.

To defend against cold and flu, Nasaleze Travel should be taken prior to exposure to a high virus risk environment. If travelling all day Nasaleze Travel should be applied 2 to 3 times over the course of the day.

Nasaleze Travel is suitable for adults, athletes, pregnant and breast-feeding women and children from 3 years.

For more information visit: [www.nasaleze.com](http://www.nasaleze.com)

Please read enclosed leaflet carefully for complete product information.

Ingredients: Inert natural cellulose powder, peppermint powder and garlic.

Keep in a cool dry place. Avoid contact with eyes. Once opened use within 6 months.

Nasaleze

Nasaleze

Nasaleze

SPRAY

TRAP & PROTECT

HELPS DEFEND AGAINST GERMS & VIRUSES

TRAVEL

POWDER SPRAY  
CLINICALLY PROVEN

CE  
Manufactured by Nasaleze Ltd, Unit 3, Woodside House, Ballalutcher Business Park, Douglas, Isle of Man, IM4 4QE, UK

Atlantic Bridge Ltd, 19 Willowdale, Woodley, Berkshire, RG5 4HJ, UK

Distributed by Nasaleze International Ltd, Nunmery Mills, Old Castletown Road, Douglas, IM2 1QA, Isle of Man

Medicines & Healthcare Products Regulatory Agency Ref. No CA007190

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The image shows the product label for Nasaleze TRAVEL 800mg. The label is green and white. At the top left, there is a factory icon and the text: "Nasaleze Ltd. Unit 3, Woodside House, Ballafletcher Business Park, Douglas, Isle of Man, IM4 4QE, UK". Below this is a box with "EC" and "REP" and the text: "Atlantic Bridge Ltd. 19 Willowside, Woodley, Berkshire, RG5 4HJ, UK". There is also a "LOT" box and an hourglass icon. In the center, the brand name "Nasaleze" is written in a stylized font, with "TRAVEL" in a shield shape below it. To the right, there is a CE mark and the text "800mg". Further right, there is a box with "Read leaflet before first use." and "Directions: Blow your nose. Apply 1 to 2 sprays into each nostril – Inhale gently while squeezing. Re-apply each time you blow your nose." Below this, it says "Keep in a cool dry place. Avoid contact with eyes. Once opened use within 6 months." At the bottom right, there is a red circular stamp with the text "CÔNG TY TNHH SV HEALTHCARE" and "THÀNH PHỐ HỒ CHÍ MINH". The stamp also contains the number "0315981074-C.T.T.N.H.C" and a signature.

Nasa-TRAVEL 11/18

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Sản xuất bởi: Nasaleze Ltd.