THACOGROUP

# MEDICAL FACE MASK

VN9500

10 PCS

### USE FOR

Preventing mechanical dust and from processing minerals including coal, iron and silica ores, cotton, powders and some other substances.

### DO NOT USE FOR

Paint & sandblasting applications or for protection against gases and vapors. This mask does not provide oxygen. Do not use in environments with oil dust.

### LIMITATIONS AND WARNINGS

1. The wearer must first be trained in proper use and compliance check before using this mask.
2. Do not use this mask when the contaminant concentration is immediately dangerous to life or health.
3. Leave the area immediately if:  
Breathing becomes difficult  
Dizziness or other suffering occurs.
4. Remove and replace the mask if it is broken or breathing becomes excessive.
5. This mask does not provide oxygen. Using atmospheric air contains less than 18% oxygen. Use only in well ventilated areas.
6. Do not change, wash, abuse or misuse this mask.
7. Do not use with other facial hair or beard or other conditions preventing a good seal between the face and sealing edge of the mask.
8. May be used to the limits specified by government regulations applicable to this product.
9. Failure to follow all instructions and warnings about the use of this mask and or not wearing this mask at all exposures may reduce the effectiveness of the mask and may lead to illness or permanent disability.

### APPLICATION

- Flu prevention
- Hospital, clinic and dentistry
- Using anti dust daily
- Beauty salon
- Industrial application

### FEATURE

- High filtration effective
- Hypoallergenic
- Without glass fibers
- Very low resistance to breathing
- No carcinogen (formaldehyde)
- No agent composition: lead, cadmium, crom 6, mercury...

### HOW TO USE

- STEP 1: Pull the ear straps
  - STEP 2: Face the mask against the chin
  - STEP 3: Pull the ear- band back to the ear and hang it on the ear until it feels comfortable
  - STEP 4: Adjust your mask to check the tightness with your face
- MASK ITEM USE 1 TIME

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