Science.
Applied to Life.™

V.A.C® & V.A.C® Veraflo™ Therapy

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3M + **KCI**

3M & KCI United by purpose.

KCI is now part of 3M. Bringing together comprehensive solutions to advance wound care and improve patient lives worldwide.

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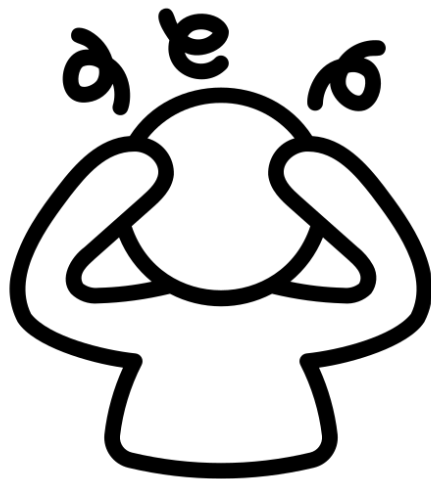
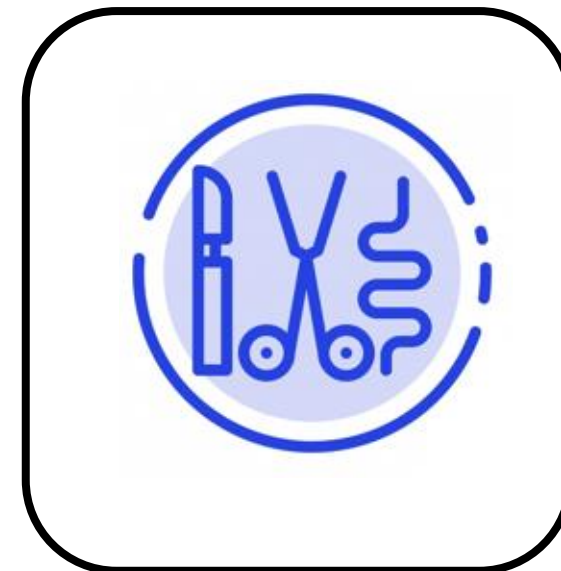
NPWT

CAN THIỆP

| Gauzes | Transparent films | Foams | Hydrogels | Hydrocolloids | Hydroconductive dressings |
|---|---|--|--|---|---|
|  <ul style="list-style-type: none"> Traditional gauzes Impregnated gauzes Wet-to-dry bandages |  <ul style="list-style-type: none"> Bi-layer or multi-layer films |  <ul style="list-style-type: none"> Traditional foams Antibacterial foams |  <ul style="list-style-type: none"> Alginate-based hydrogels Collagen-based hydrogels |  <ul style="list-style-type: none"> Internal layer based on hydrogels External layer based on synthetic polymers |  <ul style="list-style-type: none"> Multi-layer structure |

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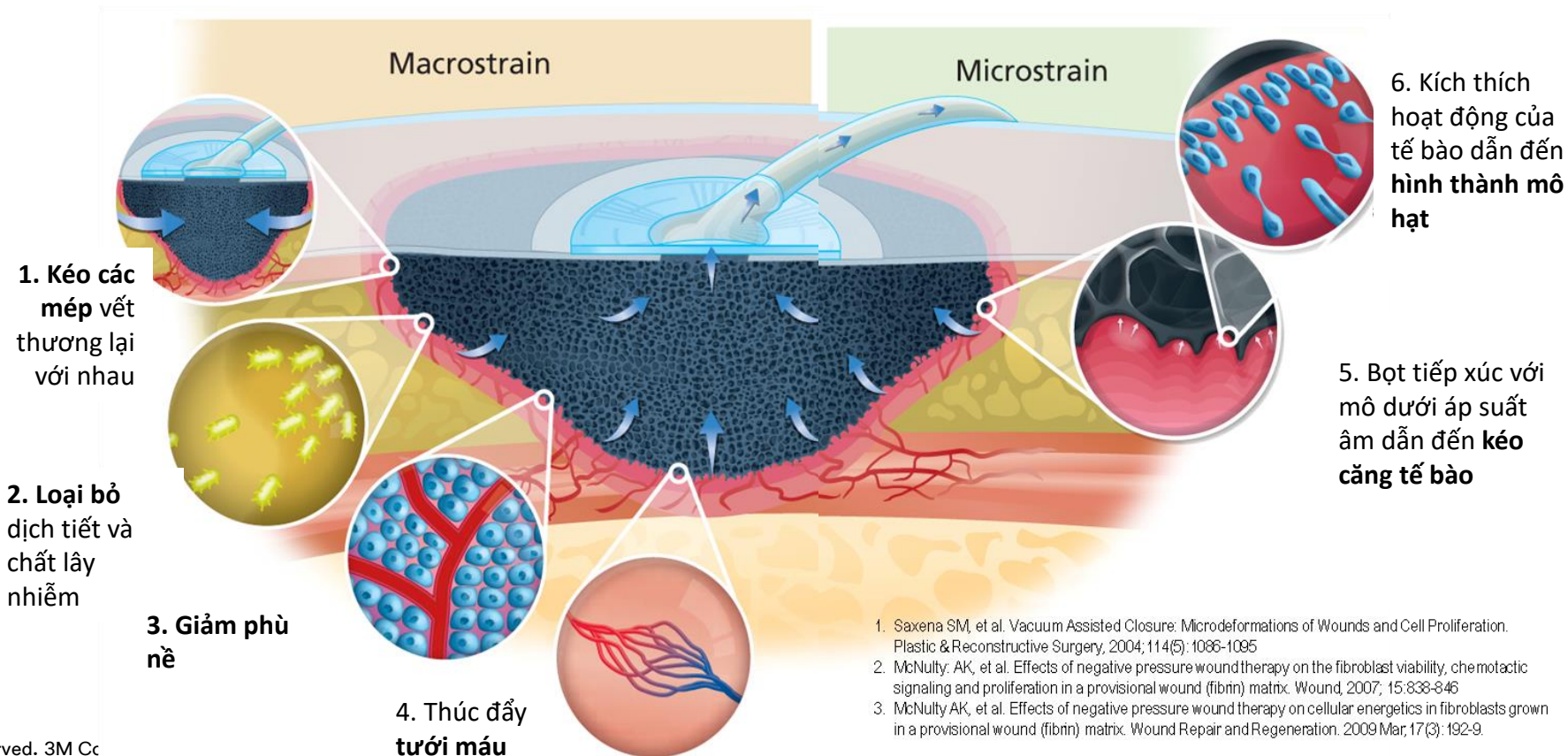
1. Saxena SM, et al. Vacuum Assisted Closure: Microdeformations of Wounds and Cell Proliferation. *Plastic & Reconstructive Surgery*, 2004; 114(5): 1086-1095
2. McNulty AK, et al. Effects of negative pressure wound therapy on the fibroblast viability, chemotactic signaling and proliferation in a provisional wound (fibrin) matrix. *Wound*, 2007; 15: 838-846
3. McNulty AK, et al. Effects of negative pressure wound therapy on cellular energetics in fibroblasts grown in a provisional wound (fibrin) matrix. *Wound Repair and Regeneration*. 2009 Mar; 17(3): 192-9.

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3M™ V.A.C.® Therapy

3M™ Prevena™ Therapy

3M™ AbThera™ Open Abdomen Negative Pressure Therapy

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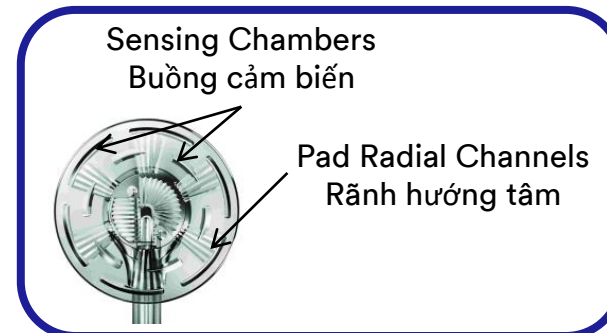
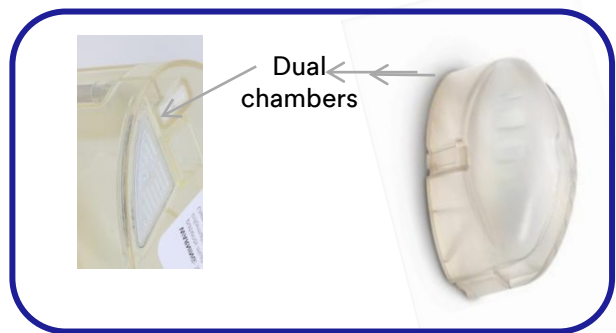
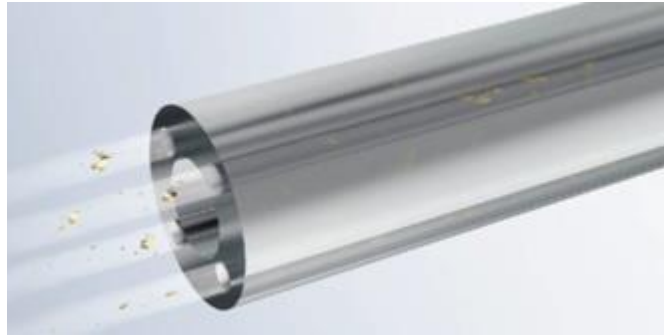
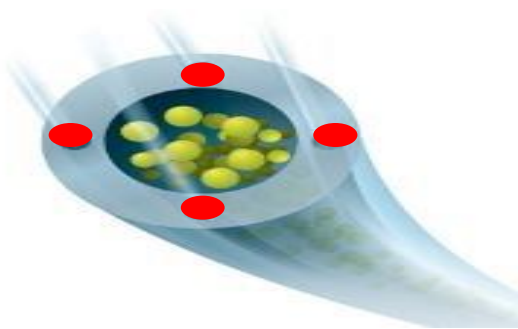
Single-lumen technology used by many providers today

KCI patented multi-lumen EASYCLEAR™ Purge Technology

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

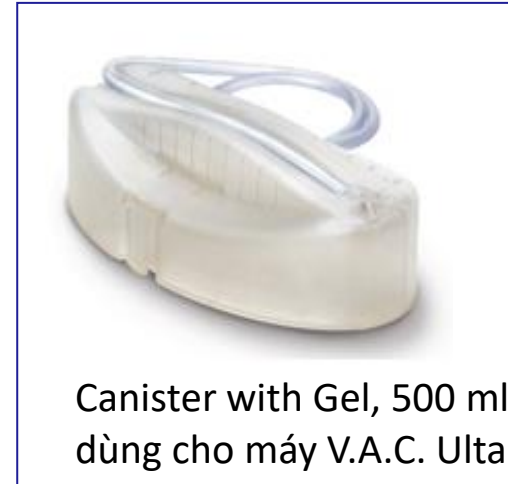
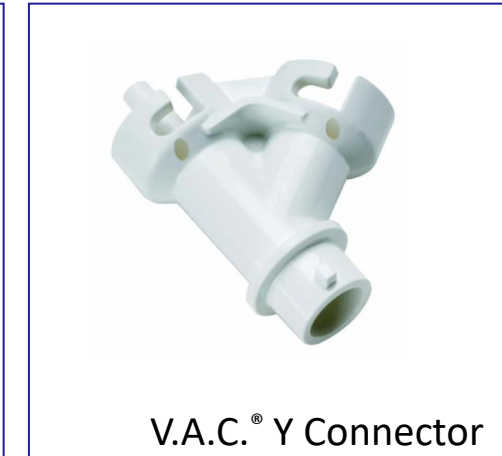
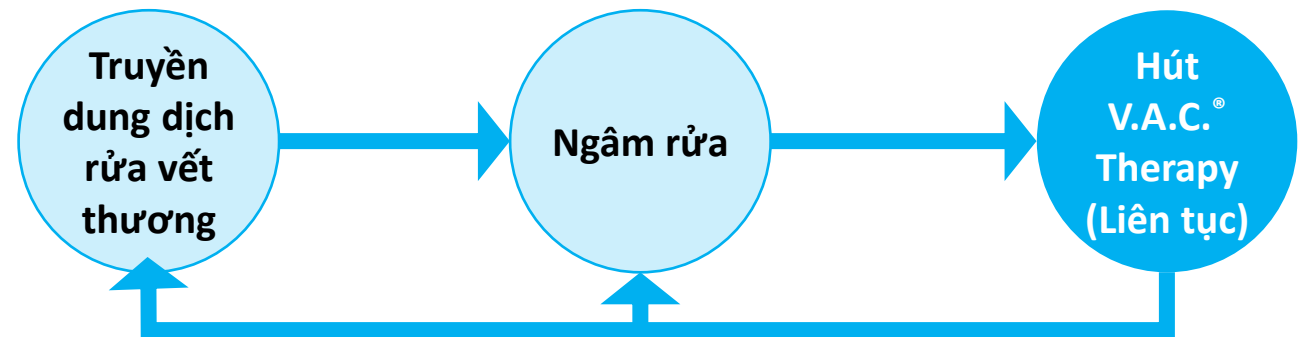
Cycle repeats for duration of therapy

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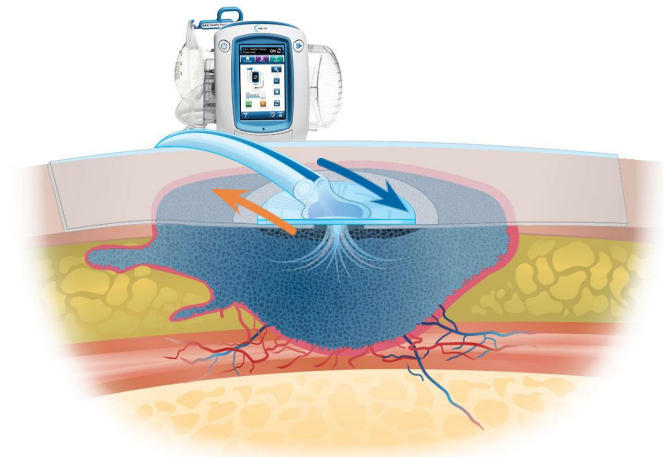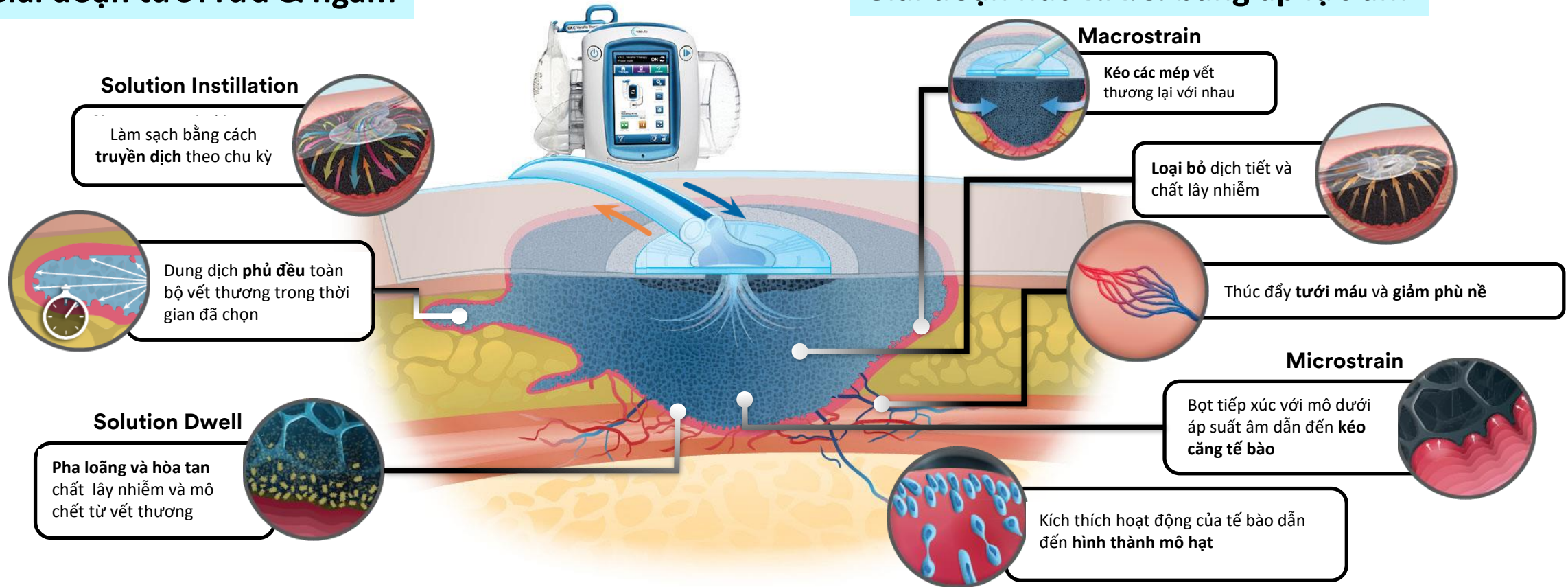*Topical solution dependent

References: 1. Rycerz AM, Slack P, McNulty AK. Distribution assessment comparing continuous and periodic wound instillation in conjunction with negative pressure wound therapy using an agar-based model. *Int Wound J.* 2013;10:214--20. DOI: 10.1111/j.1742--481X.2012.00968.x. 2. Saxena SM, et al. Vacuum Assisted Closure: Microdeformations of Wounds and Cell Proliferation. *Plastic & Reconstructive Surgery.* 2004; 114(5):1086--1095 3. McNulty AK, et al. Effects of negative pressure wound therapy on cellular energetic in fibroblasts grown in a provisional wound (fibrin) matrix. *Wound Repair and Regeneration.* 2009 Mar;17(3):192--9 4. McNulty AK, et al. Effects of negative pressure wound therapy on the fibroblast viability, chemotactic signaling and proliferation in a provisional wound (fibrin) matrix. *WOUNDS.* 2007; 15:838--846.

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>10M

wounds treated worldwide with V.A.C.® Therapy²

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3M V.A.C. Therapy negative pressure wound therapy achieves key medical evidence milestone, surpassing 2,000 peer-reviewed medical journal studies published

ST. PAUL, Minn., Oct. 13, 2022 /PRNewswire/ -- 3M Health Care's Medical Solutions Division today announced its 3M™ V.A.C.® Therapy negative pressure wound therapy (NPWT) has surpassed a clinical evidence milestone of 2,000 published, peer-reviewed medical journal studies. V.A.C. Therapy is the first and only NPWT solution to garner this number of published studies about its therapy. It is backed by more clinical data than any other brand, accounting for more than 75% of published NPWT clinical evidence.

The clinical studies have been conducted by wound care professionals worldwide and published in journals across the globe, covering a comprehensive range of wound types, wound care settings and study formats, such as case studies, economic studies, randomized controlled trials and more.

"Clinical evidence has always been a foundational element to establishing credibility for V.A.C. Therapy and our NPWT products in the wound care community," said Ronald Silverman, M.D., 3M Health Care senior vice president of clinical affairs and chief medical officer. "Published studies have also helped to promote adoption of NPWT and spur therapy innovations, including 3M™ Prevena™ Therapy for incision management, 3M™ Veraflo™ Therapy for instillation therapy for open wounds, 3M™ AbThera™ Open Abdomen Negative Pressure Therapy. Our team members in the field

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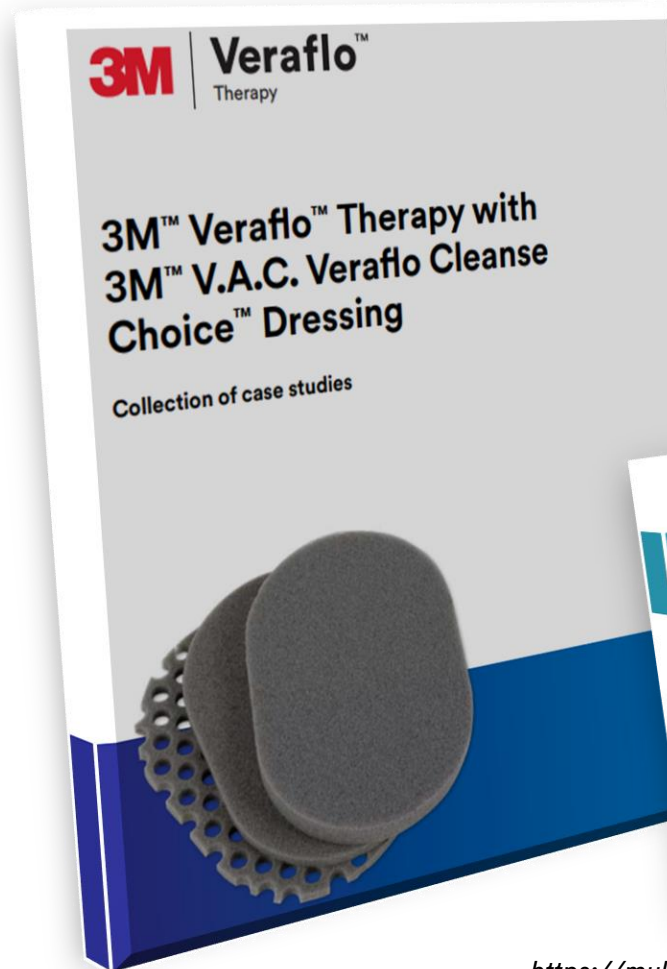
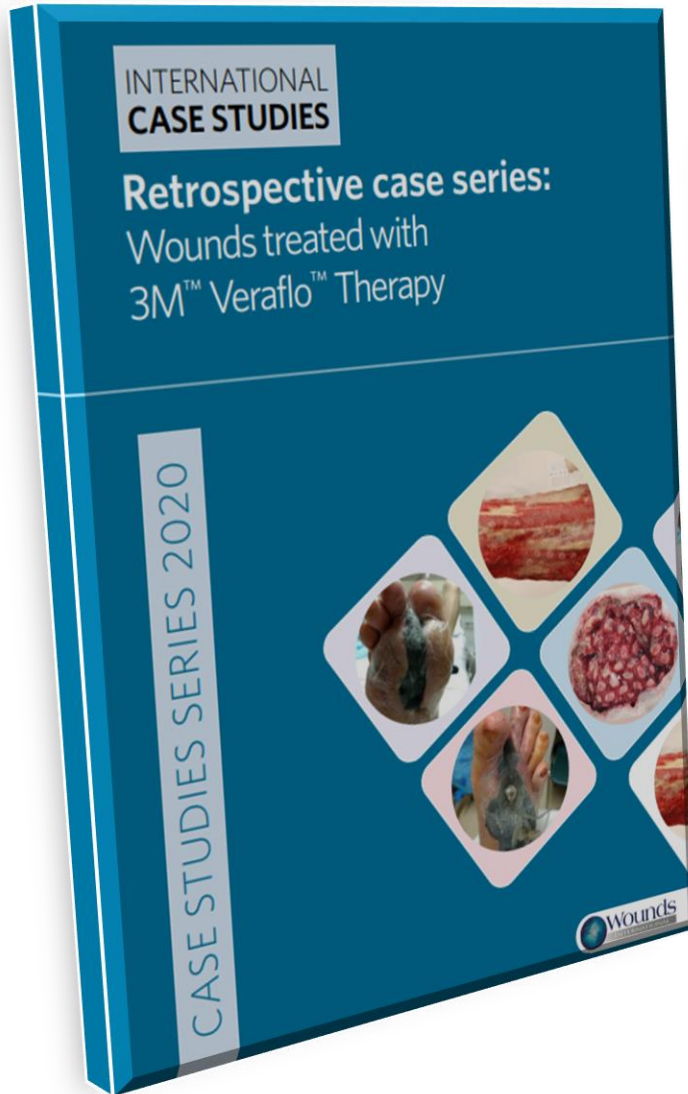
>100

peer-reviewed publications
on Veraflo Therapy

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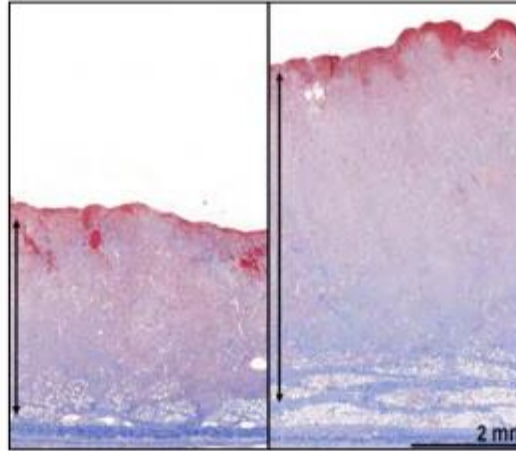
16. Lessing C, Slack P, Hong KZ, Kilpadi D, McNulty A. Negative pressure wound therapy with controlled saline instillation (NPWTi): dressing properties and granulation response in vivo. *Wounds*. 2011 Oct;23(10):309-319

Day 0: Application of non-adhering silicone dressing to protect exposed structures followed by VERAFLOR™ Therapy

Day 7: VERAFLOR™ Therapy; granulation tissue formation


Day 21: VERAFLOR™ Therapy discontinued

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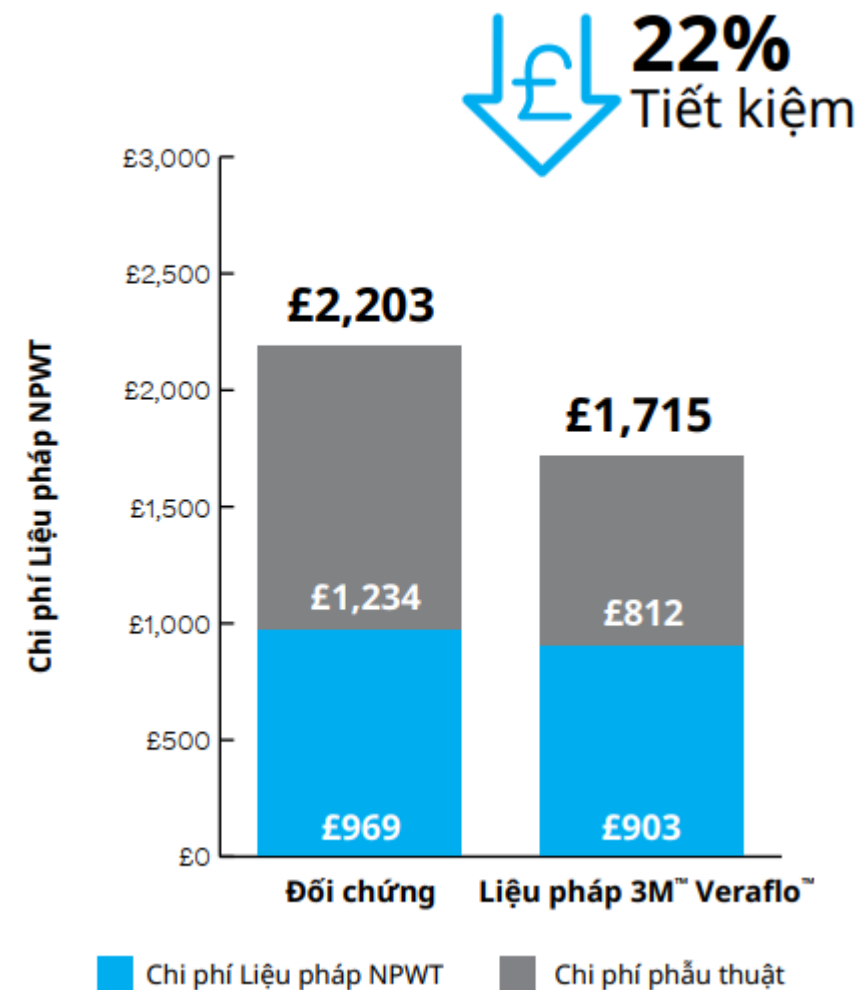17. Gabriel, Allen MD, FACS; Camardo, Mark MS; O'Rorke, Erin BS; Gold, Rebecca BS; Kim, Paul J. DPM, MS, FACFAS Effects of Negative-Pressure Wound Therapy With Instillation versus Standard of Care in Multiple Wound Types: Systematic Literature Review and Meta-Analysis, Plastic and Reconstructive Surgery: January 2021 - Volume 147 - Issue 1S-1 - p 68S-76S doi: 10.1097/PRS.0000000000007614.

18. Camardo, Mark. "Veraflo Meta-Analysis Standardized and Non-Standardized Means.", 3M Internal Report, San Antonio, Texas, 2020.

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V.A.C. VERAFLU™ Dressing

V.A.C.® GRANUFOAM™ Dressing

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
| V.A.C. (Hút áp lực âm) | VERAFLO (Hút áp lực âm + tưới rửa) |
|--|---|
| <p>Bộ V.A.C.® Granufoam™ Dressing</p>  The image shows the components of a V.A.C. Granufoam Dressing kit, including a black foam dressing, a clear adhesive pad, and various connecting tubes and ports. | <p>Bộ Veraflo™ hoặc Cleanse Choice™ Dressing</p>  The image displays two types of dressing kits: Veraflo™, which includes a clear adhesive pad with a black foam dressing inside, and Cleanse Choice™, which features a clear adhesive pad with a white mesh dressing. |
| <p>Máy + bình chứa dịch</p>  The image shows a V.A.C. pump unit with a digital display and a clear plastic reservoir for fluid collection, connected by a tube. | <p>Máy + bình chứa dịch</p>  The image shows a Veraflo pump unit with a digital display and a clear plastic reservoir for fluid collection, connected by a tube. |
| | <p>Veralink Cassette</p>  The image shows a Veralink Cassette, a clear plastic component used for fluid collection and management, connected to a tube. |

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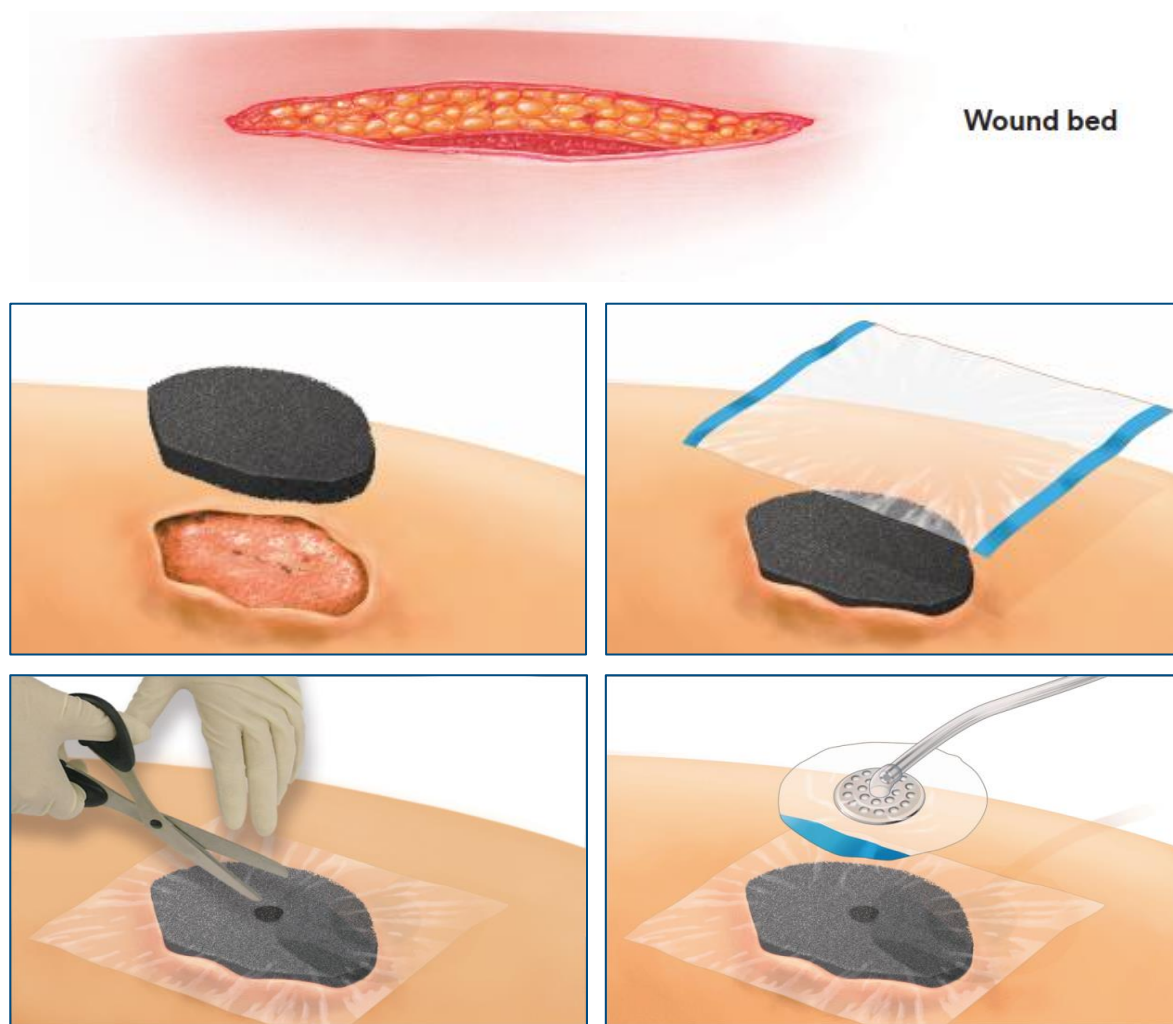
Basic Principles of Dressing Technique

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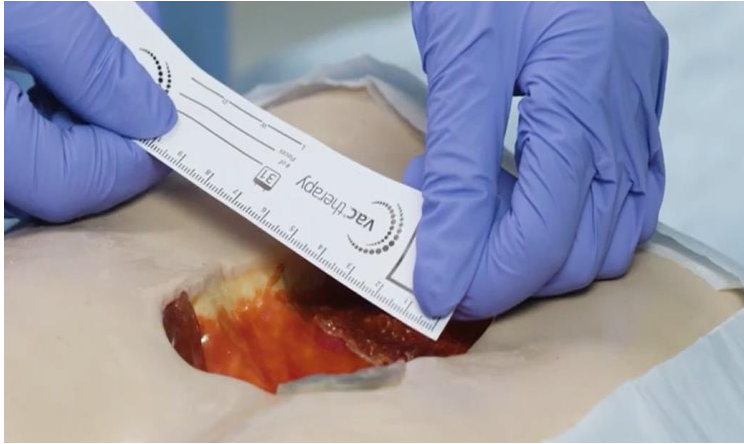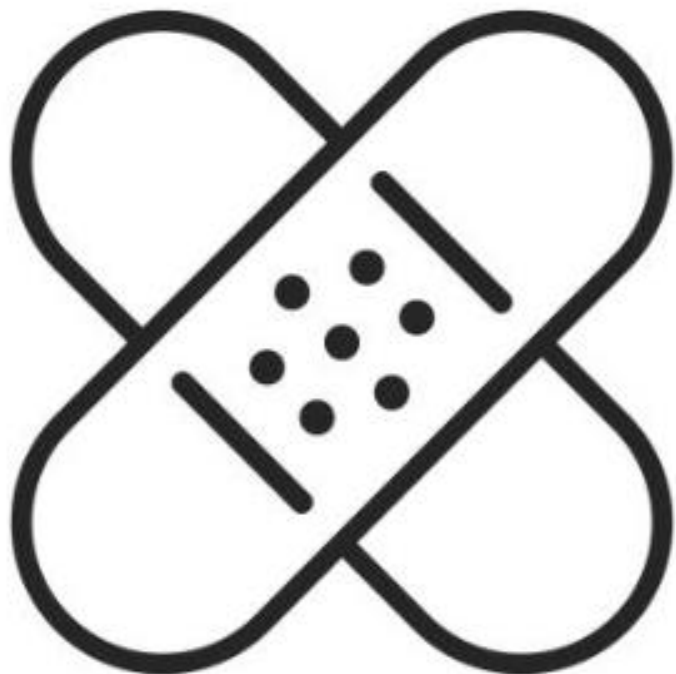
NOTE:


- Refer to Instructions For Use for COMPLETE instructions

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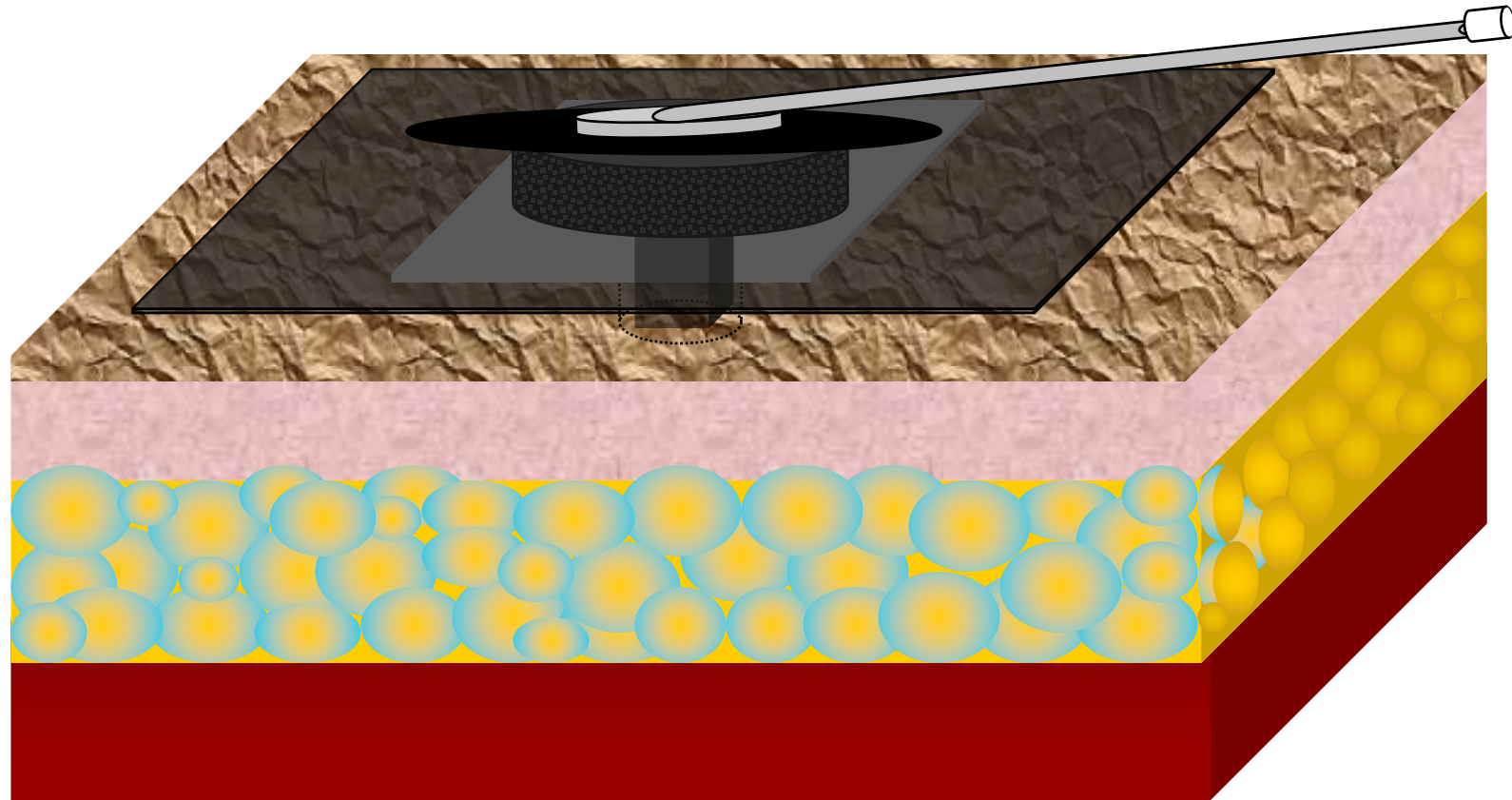
“Mushroom Technique” Wounds with Small Openings

Bridging Technique - Connecting Wounds

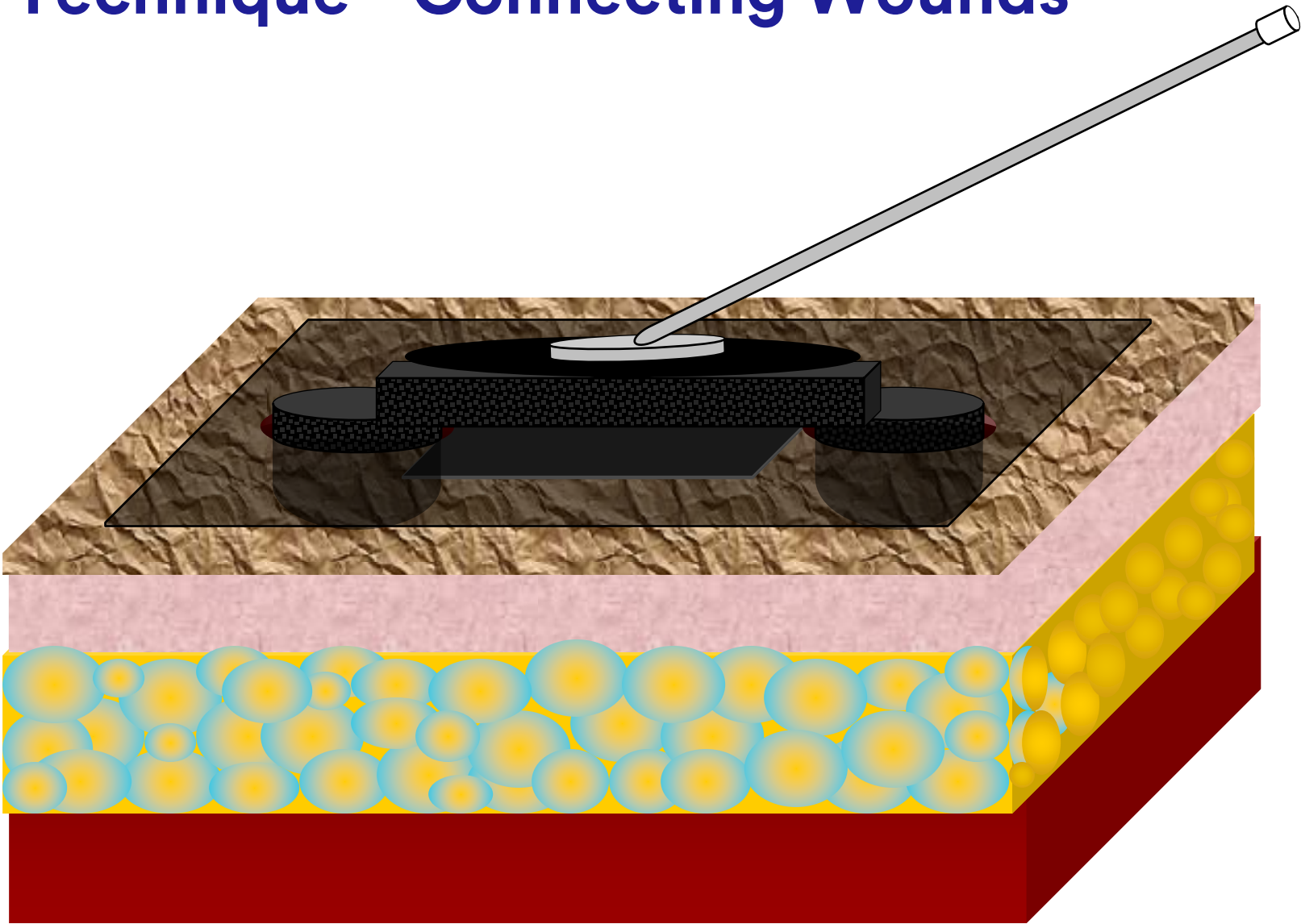Bridging Technique – Repositioning

Bridging Technique – Repositioning

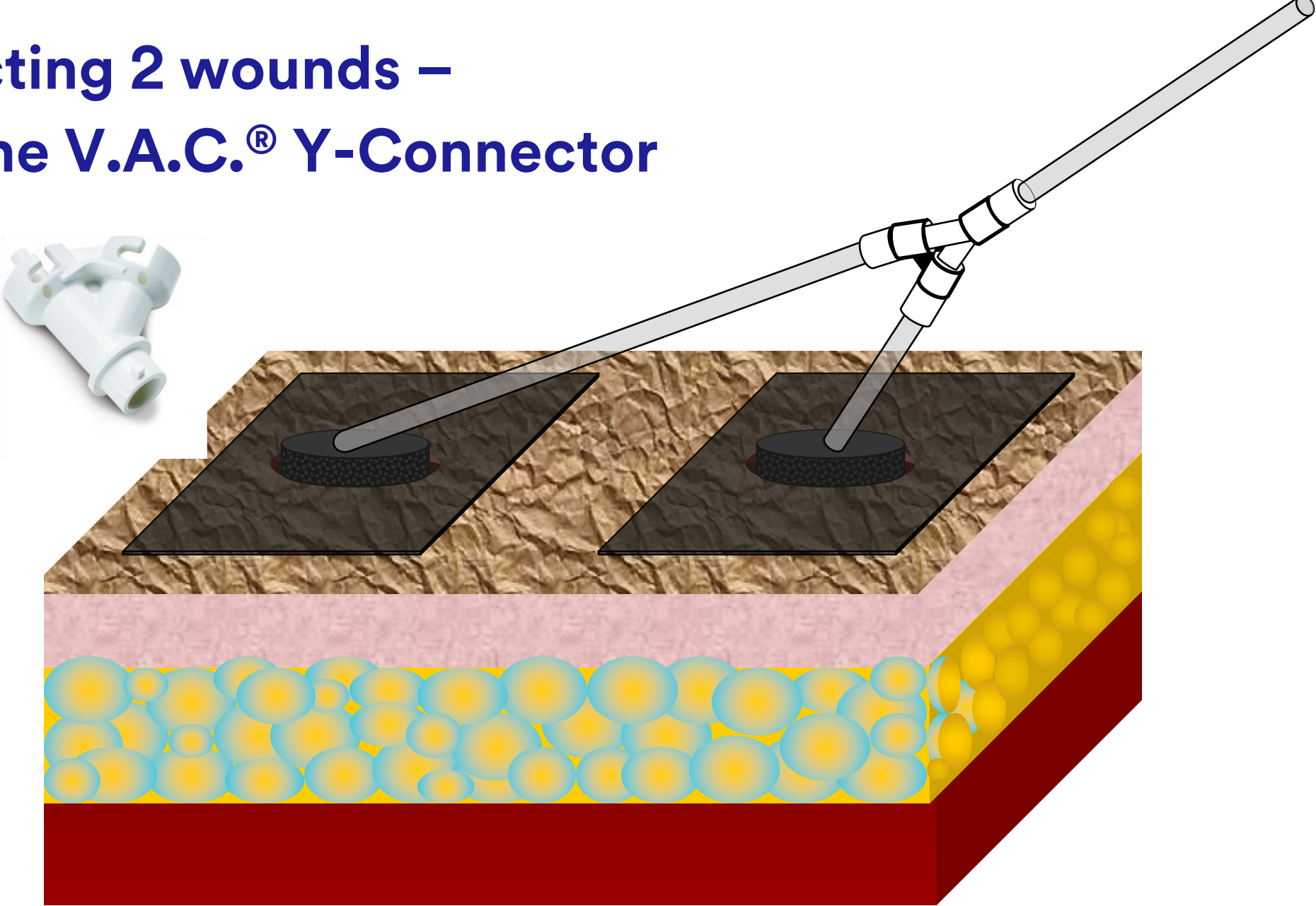
(Sacral)


(Foot)


Connecting 2 wounds – Using the V.A.C.[®] Y-Connector

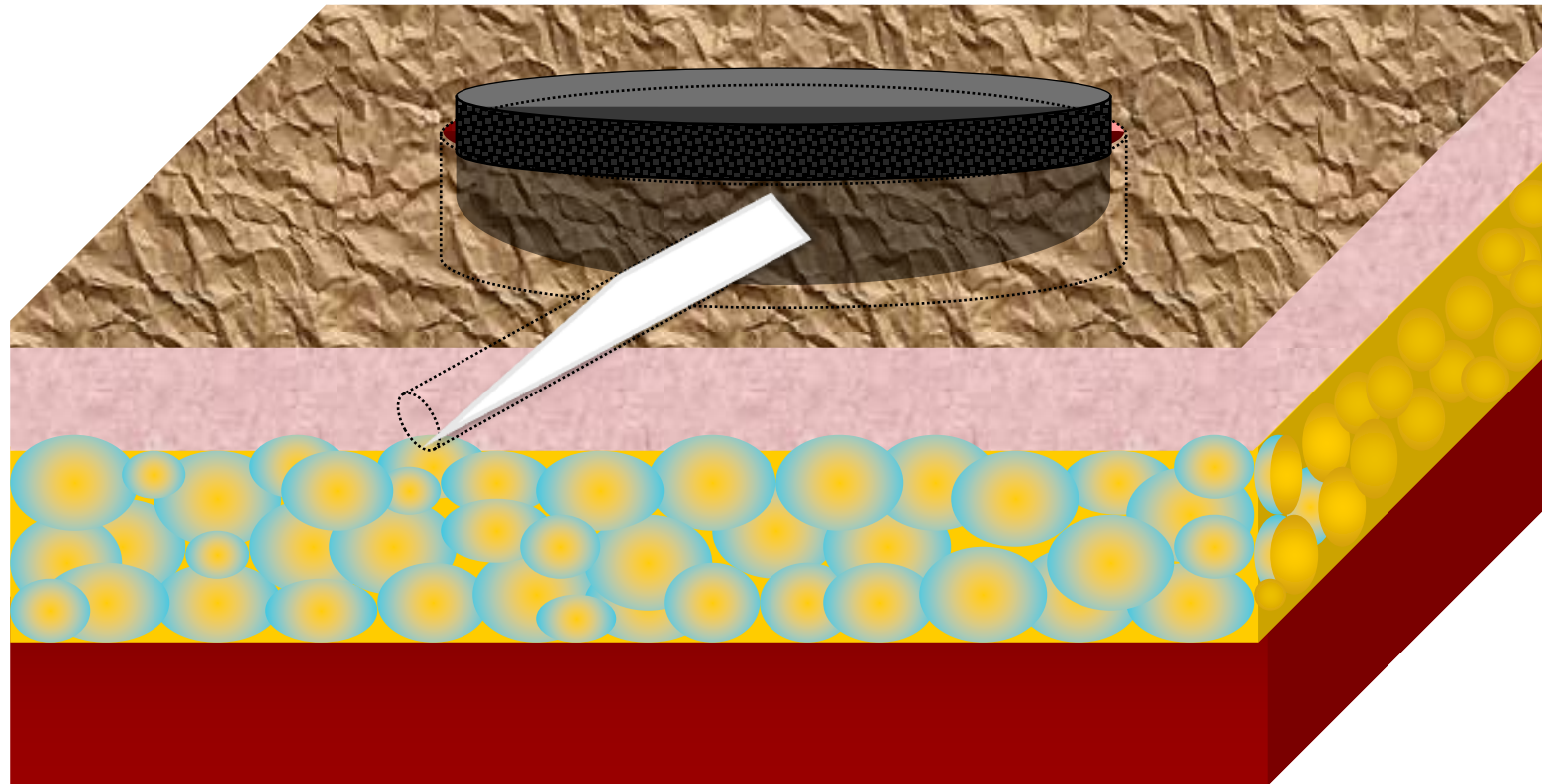Managing Tunneling in Wounds with V.A.C.® WhiteFoam

Sealing Around Hardware

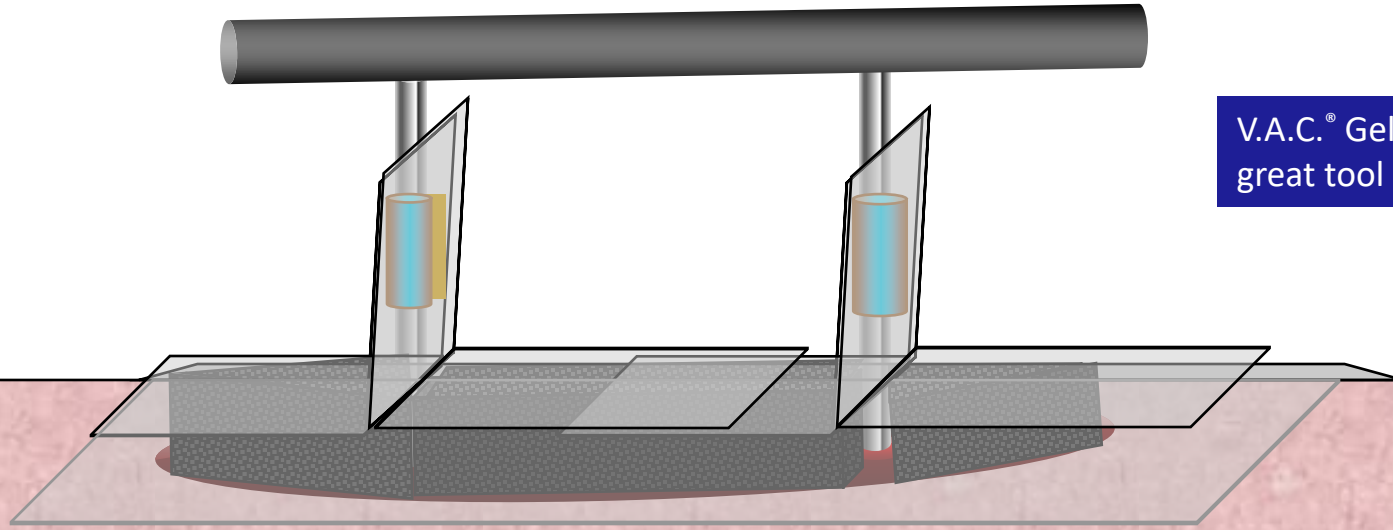
V.A.C.[®] Gel Strips are also a great tool in this area!

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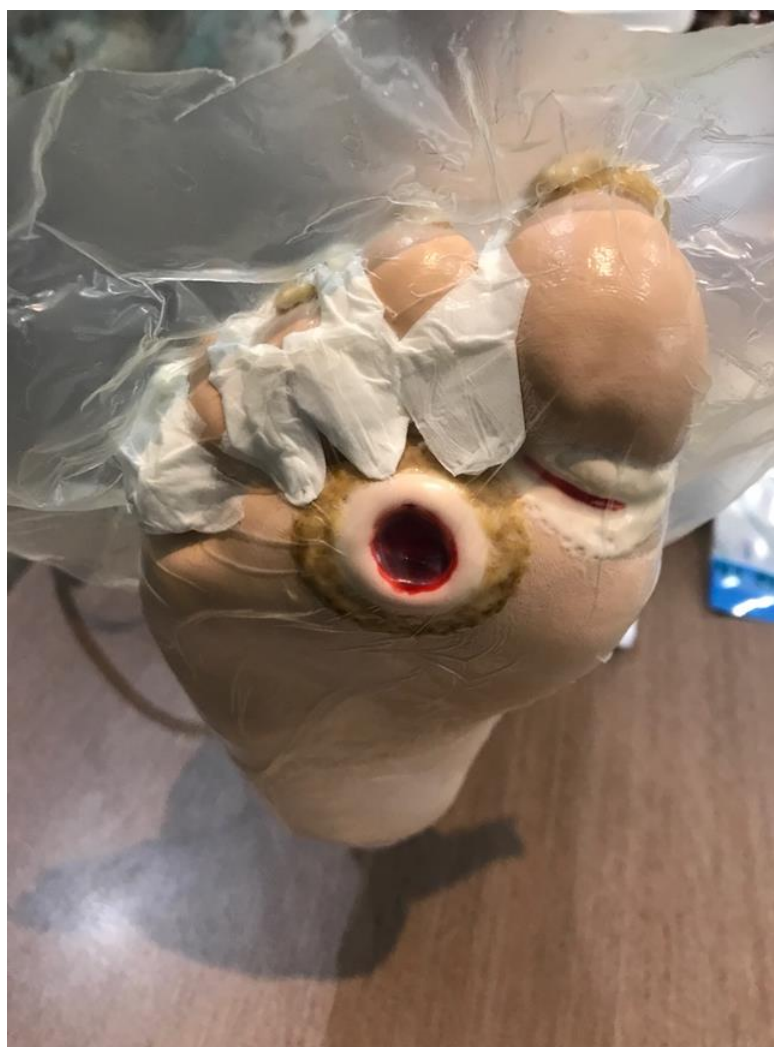This video is intended for clinicians.

Prior to use of the 3M™ V.A.C.® Therapy System, consult detailed product instructions and important prescribing information contained in the V.A.C.® Therapy Clinical Guidelines and the device specific user's guides.

The V.A.C.® Therapy System is indicated for patients with chronic, acute, traumatic, subacute and dehisced wounds, partial thickness burns, ulcers (such as diabetic or pressure), flaps and grafts.

V.A.C.® Therapy is contraindicated for:

- Patients with Malignancy in the wound
- Untreated osteomyelitis NOTE: Refer to Warnings section for osteomyelitis information.
- Non-enteric and unexplored fistulas
- Necrotic tissue with eschar present NOTE: After debridement of necrotic tissue and complete removal of eschar, V.A.C.® Therapy may be used.
- Sensitivity to silver (3M™ V. A. C.® Granufoam Silver™ Dressing only).

Additional indication and safety information specific to the V.A.C.® Therapy System is available at the end of this video. Contact your local representative if you have any questions before initiating therapy.