

HƯỚNG DẪN SỬ DỤNG

CÁNG GẤP 4 KHÚC

MODEL: YXH-1EL

I. Cấu hình kỹ thuật

1. Kích thước khi mở (L x W x H): 185 x 48 x 21cm
2. Kích thước gấp (L x W x H): 46 x 48 x 14 cm
3. Kích thước đóng gói (1 pc/bag): 50 x 48 x 15 cm
4. Trọng lượng: 7 kg
5. Trọng lượng toàn bộ: 8 kg
6. Tải trọng: <159 kg

II. Cách sử dụng

1. Mở cáng ra và đặt nó bằng phẳng trên mặt đất
2. Đặt bệnh nhân lên cáng và thắt dây an toàn

III. Áp dụng cho

Cáng gấp này chủ yếu được sử dụng để vận chuyển bệnh nhân bị gãy xương. Được chấp thuận theo thực hành phòng khám, sản phẩm này phù hợp với xe cứu thương, bệnh viện, chiến trường và phòng tập thể dục.

IV. Cảnh báo:

1. Khi mang theo người bệnh trên cáng này, đảm bảo thắt dây an toàn.
2. Khi mang theo người bệnh trên cáng này, hãy chắc chắn rằng chiếc cáng được giữ cân bằng ở vị trí nằm ngang.
3. Cần giữ gìn vải cáng và tránh các vật sắc nhọn.

V. Bảo quản:

1. Giữ cáng luôn sạch
2. Thường xuyên kiểm tra để xem có bộ phận nào bị lỏng không.

VI. Lưu trữ & vận chuyển:

1. Lưu trữ sản phẩm này ở nơi khô ráo và không bị ăn mòn.
2. Gấp cáng và đặt trong ba lô.
3. Xe vận chuyển thông thường có thể mang theo sản phẩm này.



Zhangjiagang Xiehe Medical Apparatus & Instruments Co. Ltd
FUQIAN INDUSTRIAL ZONE, ZHANGJIAGANG CITY, JIANGSU PROV., 215627,


TEL: 86-512 5817 5799 FAX: 86-512-5879 2633

<http://www.xiehe.com.cn> E-mail: allenmed@163.com

User Manual of YXH-1EL Folding Stretcher

I. Technical Features:

- 1) Product size (L x W x H): 185 x 48 x 21cm
- 2) Folded size (L x W x H): 46 x 48 x 14cm
- 3) Packing size (1pc/bag): 50 x 48 x 15cm
- 4) N.W.: 7kg G.W.: 8kg
- 5) Load bearing: less than 159kg

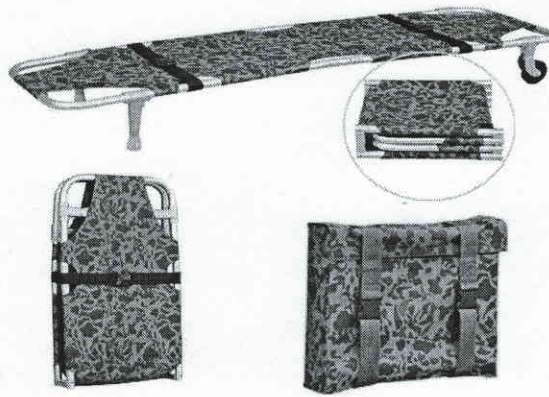

II. How to use:

1. Unfold the stretcher, and lay it flat on the ground;
2. Carry the patient on the stretcher, and fasten the safety belts.

III. Applied for:

This folding stretcher is mainly used for transporting patients with fractures. Approved by clinic practice, this item is suitable for ambulance, hospital, battlefield & gym.

IV. Cautions:

1. When carrying patients with this stretcher, make sure to fasten the safety belts.
2. When carrying patients with this stretcher, make sure the stretcher stay balanced at the horizontal position.
3. Keep the cloth cover avoid sharp objects.

V. Maintenance:

1. Keep clean as routine (incl. sterilization).
2. Often check if any parts loose or not.

VI. Storage & transportation:

1. Store this product in the place of damp proof & non-corrosion.
2. Folded the stretcher well, and put in the backpacker.
3. Common transporting vehicle can carry this product.

