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GIÁM ĐỐC

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DiaSorin Italia S.p.A.
Via Crescentino snc - 13040 Saluggia (VC) - Italy
www.diasorin.com
Tel. +39.0161.4871

Changes: Update of Legal Manufacturer Name;
Deletions: -

LIAISON® Modules (REF 319130) for 2304 tests

1. INTENDED USE

Modules required for the performance of the LIAISON® assays on the LIAISON® Analyzer.
Use only DiaSorin approved modules.

2. MATERIALS PROVIDED

6 x 64 reaction modules for 2304 tests.

6 x 64 LIAISON® Modules with 6 cavities each.

For additional details such as proper test performance of LIAISON® assays consult instructions for use of the assay to be performed.

3. HANDLING


Loading:

Place a maximum of 8 reaction modules on the transport belt as illustrated below.

Make sure that the transport belt is not in operation.

The correct position of the reaction modules is defined by a notch in the module and the guiding rail of the loading station.

Modules are for single use only.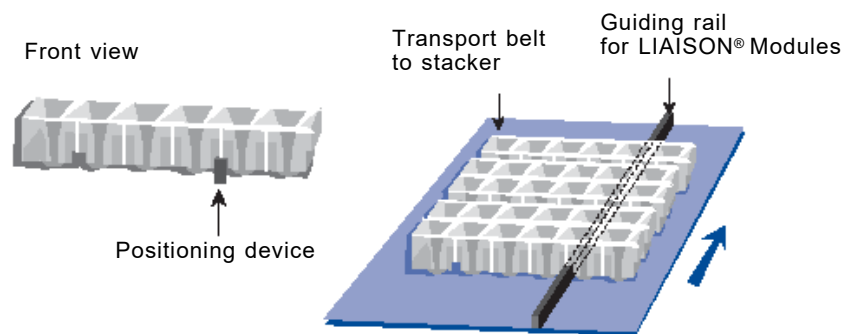

4. STORAGE

- Unopened at 15-30°C in the original box.
- Do not expose to sunlight or humidity.
- After use, reseal the box to protect the modules from dirt.
- Do not use after expiry date given on the label.

5. WARNINGS AND PRECAUTIONS FOR USERS

- Observe the normal precautions required for handling all laboratory reagents.
- Disposal of all waste material should be in accordance with local guidelines.
- For Professional Use Only.

Meaning of symbols

	In vitro diagnostic.
	Kit contents.
	Consult instructions for use.
	Protect from direct light.
	Store in an upright position.

For EU only: please be aware that any serious incident that has occurred in relation to this IVD medical device should be reported to DiaSorin and the competent authority of the EU Member State in which the user and/or patient is established.
Distributed in the U.S.A. by DiaSorin Inc., Stillwater, MN, USA.
200/008-821, 03 - 2022-07