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Website : www.wembley-med.com.vn

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ISO-PRP-006-A1_Rev.00

INSTRUCTIONS FOR USE PLATELET RICH PLASMA EXTRACTION KIT

1. Intended use

The Platelet Rich Plasma Extraction Kit (The PRP Extraction Kit) is used to extract platelet-rich plasma (PRP). After activation, platelet-rich plasma stimulates cell growth and regeneration, promotes epithelial cell proliferation, and releases growth factors that accelerate the process of wound healing and tissue regeneration. Autologous PRP is indicated for the treatment of injuries in dermatology, dentistry, orthopedics, and aesthetics.

2. Description

The PRP Extraction Kit is used to extract platelet-rich plasma from blood using a method involving negative pressure blood collection tubes and isopycnic centrifugation. This product includes vacuum blood collection tubes containing anticoagulants and other accompanying tools, all of which have been sterilized to ensure safety and effectiveness during use.

3. Precaution and Caution

- Handle all specimen samples and blood collection tools (syringes, needles, needle holders, tubes, and other blood collection devices) in standard biohazard waste containers according to the local health authority regulations.
- Provide appropriate medical care in cases of exposure to specimen samples (injuries caused by needle sticks) suspected of being contaminated with HIV (AIDS), hepatitis, and other infectious diseases.
- Overfilling or underfilling of tubes will result in an incorrect blood-to-additive ratio. This may affect the platelet extraction result.
- Do not use if the packaging is damaged or shows signs of abnormality (components in the kit are deformed, contain foreign objects, or exhibit changes in color or unexpected precipitates inside the tube, etc.).
- Do not reverse enough the specified number of times may result in suboptimal product performance.
- Do not use The PRP Extraction Kit after the expiration date.
- Wear gloves and a mask throughout the handling of The PRP Extraction Kit to reduce the risk of exposure.
- The PRP Extraction Kit must be used on the same patient from whom the blood was drawn.
- Check all product components before use.

4. Storage

The product must be stored at 4°C - 25°C (40°F - 77°F).

△NOTE: Avoid exposure to direct sunlight. Exceeding the maximum recommended storage temperature may reduce product quality.

5. Instructions

△ ALWAYS WEAR GLOVES AND MASK WHEN USING THE PRP EXTRACTION KIT

Blood Collection

- Attach the blood collection holder to the needle. Ensure the needle is securely connected and does not loosen during use.
- Apply the tourniquet and locate the vein for blood collection.
- Use an alcohol-soaked cotton ball to clean the skin around the blood collection site.

△ DO NOT TOUCH THE VENIPUNCTURE SITE AFTER DISINFECTION.

- Perform venipuncture on the patient's arm. Insert the vacuum blood collection tube into the holder until the needle punctures the rubber stopper. Remove the tourniquet immediately once blood starts to enter the tube. When the required amount of blood is collected, the flow will stop automatically.

- After collecting enough blood, carefully remove the tube from the holder and place the next tube. Immediately invert the tube gently 8-10 times to ensure proper mixing of the anticoagulant and blood. Place the tube back in the upright position and set it in the tray. **△NOTE:** Do not shake the tube vigorously as this may cause foaming or hemolysis.

- After completing the blood collection, gently withdraw the needle from the vein. Apply firm pressure to the injection site with a sterile gauze, and apply an adhesive bandage to secure and ensure safety.

Centrifugation

- Transfer the tubes to the centrifuge.
- Centrifuge the tubes at 1800G RCF for 10 minutes. **△NOTE:** Add balancing tubes if necessary.

- Using a 10ml syringe to aspirate the yellowish liquid from the top, which is platelet-poor plasma (PPP) then put it into the tube labeled PPP, leaving approximately 1-2ml of plasma at the bottom. Depending on the intended use, PPP can be activated or not activated. **△NOTE:** The needle tip can be used for ventilation to facilitate PPP/PRP collection.

- Mix well and aspirate all the remaining yellow liquid above the gel layer, which is platelet-rich plasma (PRP) into the tube labeled PRP using a 10ml syringe. **△NOTE:** Mix gently to ensure the gel layer does not get displaced.

PRP activation and sterilization

6. Contraindications

- Absolute contraindications:
 - + Platelet dysfunction syndrome.
 - + Severe thrombocytopenia.
 - + Hemodynamic instability.
 - + Sepsis.
 - + Infection at the treatment site.
- Relative contraindications:
 - + Use of NSAIDs within 48 hours prior to treatment.
 - + Corticosteroid injection at the treatment site within 1 month.
 - + Use of corticosteroids within 2 weeks.
 - + Cancer, especially blood or bone cancer.
 - + HGB level < 10 g/dl.
 - + Platelet count < 10⁹/μl.
 - + Pregnant or breastfeeding women.

7. Disposal

Medical waste has to be discarded according to current regulations, it may be segregated and treated separately if necessary.

8. Shelf life

Expiry date 12 months from the date of manufacture.

9. Symbols Glossary

	Manufacturer
	Date of manufacture
	Use-by date
	Batch code
	Catalogue number
	Sterilized using irradiation
	Sterilized using Ethylene Oxide
	Do not re-sterilize
	Do not re-use
	Do not use if package is damaged and consult instructions for use
	Keep away from direct sunlight

	Keep dry
	Temperature limit
	Country of manufacture
	Consult instructions for use
	Caution
	Stock keep unit

*NOTE: RCF Converted to RPM (Revolutions/minute)

Radius of Rotor (cm)	4	5	6	7	8	9	10	11	12	13	14	15
RPM	6400	5700	5200	4800	4500	4300	4100	3900	3700	3600	3400	3300

WEMBLEY MEDICAL FACTORY JOINT STOCK COMPANY

Lot I-10-7, D7 Street, Saigon HI-tech Park, Tang Nhon Phu Ward, Ho Chi Minh City 700000, Vietnam

Tel : +84 28 7301 3688

Email : info@wembley-med.com.vn

Website : www.wembley-med.com.vn

Made in Vietnam

IFU-PRP-006-A1_Rev.00