

Jiangsu Exor Optical Co., Ltd

No.1, Huadian Industrial Park, Development
Zone, Danyang, China Tel: +8613815498016
Email: info@honyatt.com

INSTRUCTIONS FOR USING AND HANDLING LENSES

Manufacturer: Jiangsu Exor Optical Co., Ltd

Instructions for use:

Put lenses in frame and use eyeglasses as it is.

Product is used for one time only.

Indication:

Make eyeglasses to adjust refractive errors such as myopia, hyperopia, astigmatism, and presbyopia.

Warning and Caution:

- Lenses are not suitable for some sports, especially pro-active sports.
- Lenses are weak to evaporation by hot water, hot foods, swimming, weather.
- Lenses are weak to scratch and cannot be repaired once scratched (though lenses are durable and have anti-scratch coating)

Other side effects:

- Unsuitable powers lead to bad vision, uncomfortable, or amblyopia.
- In case of myopia, wearing lenses with unappropriated powers may lead to sore or tired eyes, or experience of headaches, dizziness.

Handling:

- Do NOT soak lenses in hot water. Do NOT dry, wear glasses in sauna bathes. Do NOT leave lenses on dashboard of a car parks under sunlight. Do NOT leave lenses in motorbike trunk.

- When not in use, lenses should be cleaned and kept in case. Do NOT leave lenses with other stuff such as key... to avoid scratching and keep surface away from hitting with other objects.
- When put glasses on the table, avoid putting lenses facing downward on the surface. cleaning lenses with lenses wiping cloth. Do NOT wipe lenses with clothes or other objects.
 - Avoid using alcohol, toothpaste or strong alkali or acid detergent to clean glasses (lenses).
- Use both hands to wear and handle glasses to avoid deforming frame.
- Check glasses and tighten screws to keep lenses from falling out from its frame. Frame should be adjusted regularly to assure the fitness of glasses. Fitting frequently ensure aberration and pupil distance designed on glasses.