



## DISPOSABLE DRAINAGE BOTTLE EQUIPMENT

### (Three Bottle)

#### SCOPE OF APPLICATION:

The product is used for patients who are requested to accept closing type drainage after different kinds of thoracic operations and curing spontaneous pneumothorax, pyopneumothorax, etc.


The product has been sterilized by EO gas and kept valid within five years after 14 days as soon as sterile program be finished.

#### SPECIFIC DIRECTIONS FOR USE:

1. Unfolding the mobile bracket beneath the bottle after unsealing the packing bag. Inpouring axenic saline into bottle "B" till the water reach graduation of "8CM" . Inpouring axenic saline into bottle "C" till the water reach graduation of "12CM" . Then, Screwing down the post cap of bottle "B" .
2. Connecting bottle "A" and bottle "B" with attached 420mm length connecting tube (The inner diameter of connecting tube is 10mm).
3. Connecting one end of attached 1000mm length drainage tube (The inner diameter of connecting tube is 8mm) with drainage tube linking post on the bottle "A"(the outer diameter is 9mm). Another end of attached 1000mm length drainage tube should be linked with a 2-way stopcock and then, it can be linked with the thoracic drainage tube.
4. When minus suction be requested, connecting the suction pump with suction linking post on the bottle "C".

#### CAUTION:

1. The product should be stored in a ventilated sanitary room without corrosive gas surrounded.
2. Stop to use immediately once finding the bottle has broken or leakage etc.
3. Try to choose the exact size 2-way stopcock and thoracic drainage tube to match the attached drainage tube.
4. After drainage program finished, spill the liquid out and discharge the bottle. The bottle be designed only for single use.

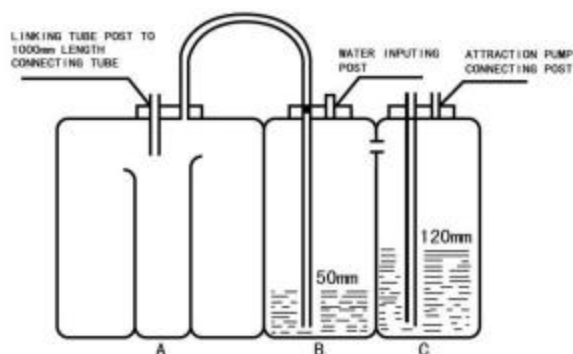

LOT. NO. : 20210928

MFG. DATE : 20210928

EXP. DATE : 20260927