

Hướng dẫn sử dụng/Instructions For Use:

- Tư thế bệnh nhân: ngồi hoặc nằm.
- Lựa chọn nẹp đúng cỡ theo chiều cao cổ.
- Lấy nẹp ra khỏi túi, mở rộng hoàn toàn các dây đai, kiểm tra dây đai ở tư thế mở.
- Đeo, lắp nẹp nhẹ nhàng, tránh gây đau cho bệnh nhân tùy theo các tư thế. Đeo nửa phía sau trước, sau đó đeo nửa phía trước ra ngoài, dán khóa Velcro sao cho đủ chắc.
- Kiểm tra cử động các khớp liên kề ở tư thế cho phép đối với từng tổn thương cụ thể theo chỉ định của bác sỹ chuyên khoa.
- Căn dặn bệnh nhân giữ tư thế cố định cần thiết, kiểm tra định kỳ thời hạn cố định cho từng tổn thương cụ thể theo chỉ định của bác sỹ chuyên khoa.
- *Posture of patient: Sitting or lying.*
- *Choose the right size according to the neck height .*
- *Take the collar out of the bag, completely pull out the straps and let them at open position.*
- *Wear the collar gently depending on the postures, avoid causing pain for patient. Firstly wear the back side then the other side at the front and stick the Velcro lock.*
- *Check the movement of nearby joint at the allowed posture for each specific injury according to doctor's advice.*
- *Patients are advised to keep necessary fixed posture, regularly check the period for immobilizing each specific injury according to doctor's advice.*

Bảng cỡ & Mã số sản phẩm/Size and product code:

Cỡ đai/Size	S	M	L
Số đo chiều cao cổ/ Collar height(cm)	7	9	11
Mã số sản phẩm/ Product code	020.OBD		

Hướng Dẫn Sử Dụng
NẸP CỔ CỨNG FIXER
Cervical Collar Fixer



Đặc điểm sản phẩm:

- Hỗ trợ, cố định cột sống cổ, đốt sống cổ.
- Có lỗ luôn ống nội khí quản.

Chỉ định:

- Sơ cứu chấn thương cột sống cổ.
- Treo cổ, căng quá mức.
- Hỗ trợ sau phẫu thuật đốt sống cổ.

Vật liệu:

- EVA, băng nhám dính.
- Khuôn nhựa.

Bảo quản:

- Để nơi khô mát.
- Có thể giặt bằng tay trong nước xà phòng, phơi khô nơi thoáng.
- Không giặt và làm khô bằng máy (không dùng thuốc tẩy).

Functions:

- Supports and immobilizes the cervical spine and cervical vertebrae.
- Includes a hole for endotracheal tube.

Indications:

- First aid for cervical spine injuries.
- Torticollis, over stretched necks.
- Support necks after cervical spine injuries.

Materials:

- EVA material, Velcro fastener.
- Plastic mold.

Preservation:

- Putting in dry place.
- Washing by hands with soap and water, drying in cool place.
- Do not wash and dry by washing machine (No using bleach).