

Hướng dẫn sử dụng/Instructions For Use:

- Tư thế bệnh nhân: ngồi hoặc đứng.
- Lựa chọn nẹp đúng cỡ theo chiều cao cổ.
- Lấy nẹp ra khỏi túi, kiểm tra dây đai ở tư thế mở.
- Đeo, lắp nẹp nhẹ nhàng, tránh gây đau cho bệnh nhân tùy theo các tư thế.
- Kiểm tra cử động các khớp liên kê ở tư thế cho phép đối với từng tổn thương cụ thể theo chỉ định của bác sỹ chuyên khoa.
- Cần dặn bệnh nhân giữ tư thế cố định cần thiết, kiểm tra định kỳ thời hạn cố định cho từng tổn thương cụ thể theo chỉ định của bác sỹ chuyên khoa.
- *Posture of patients: Sitting or standing.*
- *Choose the right size according to the neck height .*
- *Take the collar out of the bag, completely pull out the straps and let them at open position.*
- *Wear the collar gently depending on the postures, avoid causing pain for patient.*
- *Check the movement of nearby joint at the allowed posture for each specific injury according to doctor's advice.*
- *Patients are advised to keep necessary fixed posture, regularly check the period for immobilizing each specific injury according to doctor's advice.*

Bảng cỡ & Mã số sản phẩm/Size and product code:

Cỡ đai/Size	XXS	XS	S	M	L	XL
Số đo chiều cao cổ/ Neck height(cm)	5	5	6	8	10	10
Mã số sản phẩm/ Product code	030.OBD					

Hướng Dẫn Sử Dụng NẸP CỔ MỀM SOFT Foam Cervical Collar Soft



Chức năng + chỉ định:

- Dùng trong các trường hợp chấn thương cổ không cần đến cố định hoàn toàn, chỉ cố định nhẹ nhàng và hạn chế vận động.
- Có thể dùng trong cấp cứu giảm sự đau đớn trong các chấn thương cơ, gân ở cổ và đốt sống cổ.
- Sau phẫu thuật mạch máu, cân cơ vùng cổ.

Vật liệu:

- Mút Foam PU, nhẹ và thoáng khí.
- Vải, băng nhám dính.

Bảo quản:

- Để nơi khô mát.
- Có thể giặt bằng tay trong nước xà phòng, phơi khô nơi thoáng.
- Không giặt và làm khô bằng máy (không dùng thuốc tẩy).

Functions and Indications:

- *Used for patients with neck injuries but not totally immobilized, just fixed gently and limited movement.*
- *Can be used in emergency to reduce pain in cervical muscles, tendons and spines.*
- *Used after vein, muscle surgeries.*

Materials:

- *Soft PU foam, light and airy foam.*
- *Fabric, Velcro fastener.*

Preservation:

- *Putting in dry place.*
- *Washing by hands with soap and water, drying in cool place.*
- *Do not wash and dry by washing machine (No using bleach).*