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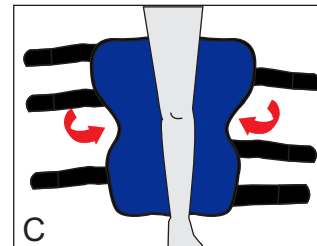
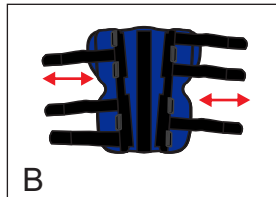
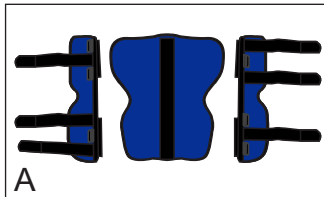
Functions:

- Supports and stabilizes the knee joint and lower leg; restricts movement as prescribed.
- Assists in immobilizing the area around the knee joint and lower leg.

Indications:

- First aid immobilization for injuries to the knee joint and periarticular region.
- Support, immobilization, and motion restriction following knee surgery.
- Support and immobilization for thigh and lower leg injuries as directed by a specialist physician.

Materials:

- Cotton fabric, tricot fabric, PU foam.
- Velcro straps, anodized aluminum alloy splints.

Preservation:

- Putting in dry place.
- Washing by hands with soap and water, drying in cool place.
- Do not wash and dry by washing machine (No using bleach).

Instructions For Use:

- Patient position: lying or sitting with knees stretched.
- Brace check: Select the brace size suitable for the patient. Remove the brace from the bag, disassemble and check the 3 pieces of the brace, leaving the straps open.
 - Step 1: Check whether the positions of two side splints fit the leg properly. The side splints can be adjusted by moving outward or inward so that they align with two sides of the leg (The manufacturer packs the product in medium size). (picture A+B)
 - Step 2: Place the brace behind the knee so that the large panel fits the popliteal fossa following its natural curve. (picture C)
 - Step 3: Secure the brace to the injured area in the following order: first the middle belt strap, then the upper and lower straps. Tighten the straps gradually, ensuring that they are secure but do not cause pain or restrict circulation. (picture D)