

Hướng dẫn sử dụng/Instructions For Use:

Lựa chọn kích cỡ phù hợp với số đo vòng ngực, kiểm tra sản phẩm trước khi sử dụng rồi tiến hành các bước sau:

- Tư thế bệnh nhân: ngồi, hai tay chống mạn sườn, uốn ngực.
- Lấy đai ra khỏi túi, mở rộng, kiểm tra các dây đai ở tư thế mở.
- Đeo, lắp đai nhẹ nhàng, tránh gây đau cho bệnh nhân tùy theo các tư thế. Cố định từng nấc từ lỏng đến chặt (đủ chắc, không gây đau hay chèn ép).
- Kiểm tra cử động các khớp liên kế ở tư thế cho phép đối với từng tổn thương cụ thể theo chỉ định của bác sỹ chuyên khoa.
- Cần dặn bệnh nhân giữ tư thế cố định cần thiết, kiểm tra định kỳ thời hạn cố định cho từng tổn thương cụ thể theo chỉ định của bác sỹ chuyên khoa.

Choose suitable size with chest circumference, check the product before use and follow these steps:

- Posture of patient: sitting, hands on side of flank, puffing out chest.
- Remove the belt from the bag, expand, check the straps in the open position.
- Wear; use belt gently, avoid causing pain to the patient depending on the postures. Fix each step from loose to tight (strong enough, no pain or pinching).
- Check the movement of adjacent joints in the allowed postures for each specific injury as directed by a specialist.
- Periodically check the fixed term for each specific injury as assigned by a specialist.

Bảng cỡ & Mã số sản phẩm/Size and product code:

Cỡ đai/Size	1	2	3	4	5
Số đo vòng ngực/ Chest circumference (cm)	34-41	41-48	48-53	53-58	58-63
Mã số sản phẩm/ Product code	141				

Hướng Dẫn Sử Dụng ĐAI SỐ 8 TRẺ EM *Figure of Eight for Children*



**Chức năng:**

- Neo ép, cố định xương đòn.

Chỉ định:

- Cố định chấn thương gãy xương đòn, trật khớp cùng đòn.
- Vỡ xương bả vai, trật khớp ức - đòn.
- Teo cơ delta, cơ ức - đòn - chũm.

Vật liệu:

- Vải cotton, mút xốp.
- Khóa Velcro.

Bảo quản:

- Để nơi khô mát.
- Có thể giặt tay bằng nước có xà phòng, phơi khô nơi thoáng mát.
- Không giặt và làm khô bằng máy (không dùng thuốc tẩy).

**Functions:**

- Press and fix clavicle collar-bone.

Indications:

- Fix clavicle collar-bone fractured trauma and dislocated.
- Broken shoulder-blade, dislocated breastbone.
- Delta atrophy, sternocleidomastoid muscle.

Materials:

- Cotton, foam padded.
- Velcro lock.

Preservation:

- Putting in dry place.
- Washing by hands with soap and water, drying in cool place.
- Do not wash and dry by washing machine (No using bleach).