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Functions:

- Fix areas around the knees, thighs and lower legs.

Indications:

- Provide first aid after injury to joint and around the knee joint.
- Support and fix after injury and surgery around the knee.
- Support and fix for thigh and leg injuries as directed by specialists.

Materials:

- Mesh fabric and PU foam.
- Velcro lock.
- Aluminum alloy splint bent to the body curve, stiffness-adjusted, and anodized on the surface.

Preservation:

- Putting in dry place.
- Washing by hands with soap and water, drying in cool place.
- Do not wash and dry by washing machine (No using bleach).

Instructions For Use:

- Posture of patient: lying or sitting with knees stretched.
 - Remove product from the bag, wide-opening, check product while opening all straps.
 - Wear, install the brace gently, avoid causing pain to the patient depending on the postures. Fix the middle strap first, then go up and go down, fix each step from loose to tight (strong enough, no pain or pinching).
 - Check the movement of adjacent joints in the allowed posture for each specific injury as directed by a specialist.
 - Recommend patient to maintain fixed posture necessarily, periodically check the fixed term for each specific injury as directed by a specialist.
- Note: The splints on the product can be taken off, depending on the specific patient's case, the doctor can adjust suitably by curving the splints as required.