14,5 cm

21 cm

Nasal spray

PROMAX

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(Promax iNO 240, Promax iNO 150, Promax iNO 60, Promax iNO Kids 150, Promax iNO Kids 60)

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Read all of this leaflet carefully before use.

PROMAX® iNO is a set of medical equipment class A that uses dual device technology and mist nozzle to create nitric oxide gas in the nasal cavity (Inhale Nitric Oxide - iNO) immediately after pressing the spray, helping treat sinus diseases and helping conserve antibiotics.

COMPOSITION

The main ingredient is Nitric Oxide.

Nitric oxide nasal spray is made up of a dual bottles:

The composition of the solution in bottle A:

Sodium nitrite USP 0.5% w/v

Benzalkonium chloride USP 0.02% w/v

The composition of the solution in bottle B:

Citric acid USP 0.2% w/v

Sodium chloride USP 0.9% w/v

Benzalkonium chloride USP 0.02% w/v

DOSAGE FORM

Nasal spray solution

PHYSIOLOGY AND MECHANISM OF ACTION

Nitric oxide (NO) is a natural endogenous substance that plays an important role and has many applications in medicine. In the body, NO is converted from L-arginine to L-citrulline with the catalysis of the enzyme NOS (Nitric Oxide Synthase)... In the upper respiratory tract, the maxillary sinus is the body's largest nitric oxide reservoir. NO in the nose has the effect of regulating air flow and non-specific protection, destroying pathogens through the nose and sinuses. Epidemiological reports show that over 90% of the causes of sinus diseases are viruses.

A randomized, double-blind phase 3 clinical study at 20 centers showed that nitric oxide nasal spray killed over 99% of viruses transmitted through the nasal sinuses in 48 hours and reduced the viral load 16 times more than deep sea water. At low concentrations, nitric oxide breaks down biofilm (a layer of membrane where certain types of microorganism clusters together that make antibiotics hard to penetrate). Biofilm bacteria cause persistent cases of sinusitis and reduce the effectiveness of antibiotics and increase antibiotic resistance. Nitric oxide has a wound-healing effect through its central role in regulating three key parts of the healing process: vascular homeostasis, anti-inflammation, and antibacterial effects. Nitric oxide also helps increase blood flow to sinus tissue, helping to quickly eliminate inflammatory cell fluids while increasing the activity of drugs that need to reach sinus tissue. Inadequate nasal nitric oxide levels are also associated with ciliadysfunction in the nasal mucosa. When using nitric oxide, cilia cells in the sinuses will be activated after 24 hours, helping to increase the elimination of pathogens and help make the airways more fresher.

Prevent superinfection, Synergistic with antibiotics

INDICATION

PROMAX® iNO is used in the supportive treatment of cases of acute rhinosinusitis due to viral infection, acute rhinosinusitis post viral infection, chronic rhinosinusitis with persistent (recurrent) infection while using antibiotics.

PROMAX® iNO helps reduce symptoms of flu, colds, itchy nose, runny nose, rhinitis, and post-sinus surgery.

PROMAX® iNO helps prevent viruses from entering the body through the nose shall increase the risk of superinfection in people with primary and secondary immunodeficiency, people in closed environments, people who travel frequently, or people with high exposure to viruses and pathogens through the nose and sinuses.

SUBJECTS OF USE

Adults and children over 2 years old (know how to inhale when spraying into the nasal cavity)

DOSAGE

The usual dose is 2 sprays in each nostril 2 - 3 times a day or follow the doctor's instruction.

HOW TO USE AND DURATION OF USE

- Clean your nose before use.
- Tilt your head slightly back. Inhale and quickly press the device once to spread the spray solution into the nasal cavity (You can use 2 hands to ensure the solution comes out evenly from the 2 bottles).
- Use period is 14 days during an acute duration
- Use prophylactically for 07 days whenever you have flu symptoms, itchy nose, stuffy nose, runny nose, rhinitis or when in an environment with poor air circulation (flying by plane, train, car, air conditioner...).
- Long-term prophylactic use for immunodeficiency patients.

CONTRAINDICATIONS

Do not use if you are allergic to any of the ingredients in the product.

USE FOR PREGNANCY AND LACTATION WOMEN

Consult doctor/ pharmacist before use.

PROBLEM ADVERSE EFFECTS

No adverse effects have been observed when used at the recommended dose.

PACKAGING:

Promax iNO 240 (240 sprays); Promax iNO 150 (150 sprays); Promax iNO 60 (60 sprays); Promax iNO Kids 150 (150 sprays); Promax iNO Kids 60 (60 sprays).

STORAGE CONDITIONS AND SHELF LIFE

Storage: Keep away from direct sunlight and temperature not exceeding 30°C. Close the lid tightly immediately after use. Do not store in the freezer.

Shelf life: 36 months from date of manufacture

MANUFACTURER:

CENTRAL PHARMACEUTICAL JOINT STOCK COMPANY 3
115 Ngo Gia Tu, Hai Chau Ward, Da Nang City, Vietnam

REGISTER:

AVASURE LIFE SCIENCE JOINT STOCK COMPANY
4th floor, Anh Minh Building, 36 Hoang Cau, O Cho Dua Ward, Hanoi, Vietnam

DISTRIBUTOR:

ALPHA FRANCE PHARMA JOINT STOCK COMPANY
7th floor, 60 Nguyen Van Thu, Tan Dinh Ward, Ho Chi Minh city, Vietnam

☎ 1900 99 88 09

🌐 www.pro-max.com.vn